

2023

COMPUTER
CONFIGURATION &

Downloading Dental Software SHINING 3D



Lentil Evergreen

Copyright ©2020-2022 SHINING 3D

All rights reserved.


January 2023



SHINING 3D®

www.shining3ddental.com

SYSTEM CONFIGURATION

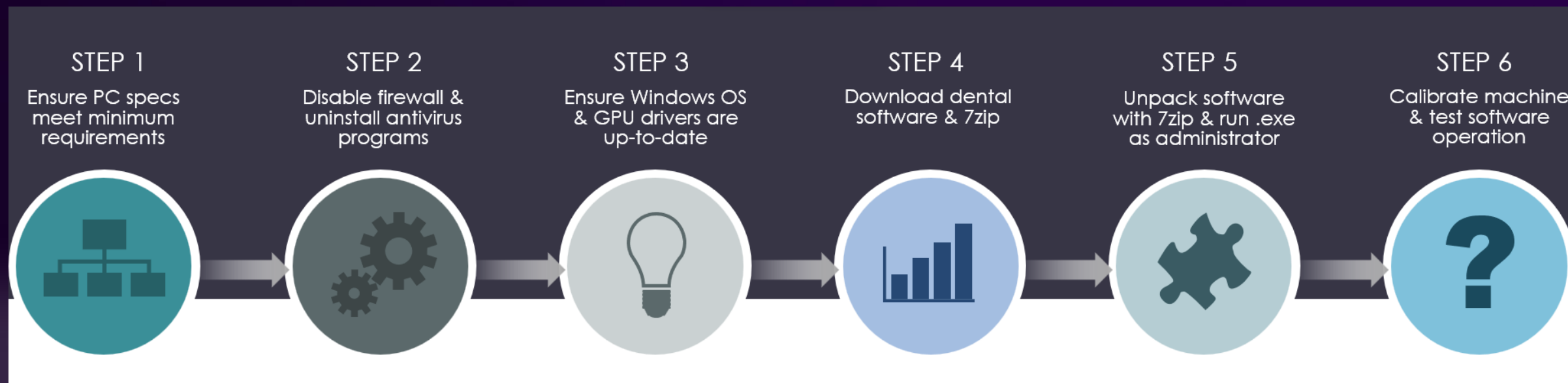

STEP 1: MEET MINIMUM PC REQUIREMENTS

DentalScan

IntraoralScan

AccuWare

DENTALSCAN MINIMUM SYSTEM REQUIREMENTS

Item	Requirements
Operating System	Windows 10, 11, Windows 11 Pro
Memory (RAM)	16GB or above
Storage	• 32GB for scanning (minimum 64GB for DS-MIX & DS-EX Pro(H)) • 1TB for storage (minimum 512GB for DS-MIX & DS-EX Pro(H))
Display	27" or larger (2560x1440 or above)
Graphics	• NVIDIA GeForce RTX 3060 or above • AMD Radeon RX 6600 or above
Network	• Ethernet connection required • 100Mbps or above
USB Ports	• 1x USB 3.0 port • 1x USB 3.0 port (for DS-MIX & DS-EX Pro(H))
Power supply	AC input: 100-240V, 1.5A, 50-60Hz

Recommended equipment

- Intel Core i7-8700
- 16GB RAM
- 32GB for DS-MIX & DS-EX Pro(H)
- 1TB for storage (minimum 512GB for DS-MIX & DS-EX Pro(H))



INTRAORALSCAN MINIMUM SYSTEM REQUIREMENTS

Item	Requirements
Operating System	Windows 10 Professional (64-bit) or above
Memory (RAM)	16GB or above
Storage	• 32GB for scanning (minimum 64GB for DS-MIX & DS-EX Pro(H)) • 1TB for storage (minimum 512GB for DS-MIX & DS-EX Pro(H))
Display	27" or larger (2560x1440 or above)
Graphics	• NVIDIA GeForce RTX 3060 or above • AMD Radeon RX 6600 or above
Network	• Ethernet connection required • 100Mbps or above
USB Ports	• 1x USB 3.0 port • 1x USB 3.0 port (for DS-MIX & DS-EX Pro(H))
Power supply	AC input: 100-240V, 1.5A, 50-60Hz


Recommended equipment

- Intel Core i7-8700
- 16GB RAM
- 32GB for scanning (minimum 64GB for DS-MIX & DS-EX Pro(H))
- 1TB for storage (minimum 512GB for DS-MIX & DS-EX Pro(H))



ACCUREWARE MINIMUM SYSTEM REQUIREMENTS


Item	Requirements
Operating System	Windows 10 Professional (64-bit), Windows 11
Memory (RAM)	16GB
CPU	Intel Core i5-8500 (9M Cache, up to 4.10 GHz)
GPU	NVIDIA GTX 1050 Ti
SSD	C:\ Drive (after installation) >10 GB
Power	AC input: 100-240 V, 360 W



ACCUREWARE MINIMUM SYSTEM REQUIREMENTS

AccuWare

- OS: Windows 10 Professional (64-bit), Windows 11
- RAM: 16 GB
- CPU: Intel Core™ i5-8500 (9M Cache, up to 4.10 GHz)
- GPU: NVIDIA GTX 1050 Ti
- SSD: C:\ Drive (after installation) >10 GB
- Power: AC input: 100-240 V, 360 W



ACCUREWARE MINIMUM SYSTEM REQUIREMENTS

AccuFab L4D & D1S

- Temperature: 23°C ± 2°C
- Humidity: 50% ± 10%
- Workspace: A neat, stable, and level platform



DENTALSCAN MINIMUM SYSTEM REQUIREMENTS

- OS: Windows 10 Professional (64-bit), Windows 11
- RAM: 16 GB, **32 GB for DS-MIX & DS-EX Pro(H)**
- CPU: Intel Core™ i7-8700
- GPU: NVIDIA GTX 1060 (6 GB RAM)
- USB: USB 3.0 port + additional USB port
- SSD: EXE location ≥2 GB, DentalOrder location ≥20 GB
- Power: AC input: 100-240 V, 1.5 A, 50-60 Hz
- Network: Internet connection required



INTRAORALSCAN MINIMUM SYSTEM REQUIREMENTS

- OS: Windows 10 Professional (64-bit), Windows 11
- RAM: 16 GB
- CPU: Intel Core™ i7-8700
- GPU: NVIDIA RTX 3060, **not including RTX 3050**
- USB: ≥2 USB 3.0 port (type A)
- SSD: Total ≥256 GB, EXE location ≥8 GB, DentalOrder location ≥20 GB
- Resolution: ≥1920 x 1080
- Power: AC input: 100-240 V, 1.0 A, 50-60 Hz
- Network: Internet connection required



DENTALSCAN MINIMUM SYSTEM REQUIREMENTS

System Requirements for using DentalScan V3.0.0.9 or later

Item	Requirments
Operating System	Windows 10 1809, Windows 11
Memory (RAM)	16GB or above
<ul style="list-style-type: none">Note:For scanning impressions using EX Pro(H) or MIX scanners, A minimum of 32GB RAM is required.	
Display card (Nvidia discrete graphics card)	Nvidia GTX1060 (RAM 6GB) or above
CPU	Intel(R) Core(TM) i7-8700 or above (Intel processor only)
Available Space	<ul style="list-style-type: none">Software installation location with a minimum available space of 2GBOrder storge location with a minimum available space of 20GB
Network	Internet connection is required.
USB Ports	<ul style="list-style-type: none">scanner needs one USB 3.0 port;scanner dongle needs one USB port with no preference
Power supply	AC input: 100-240V~,1.5A 50-60Hz

Environment requirment

- Work temperature: 10°C ~ 30°C
- Keep the scanner away from bright light
- Set the scanner steady without vibration



DENTALSCAN MINIMUM SYSTEM REQUIREMENTS

- OS: Windows 10 Professional (64-bit), Windows 11
- RAM: 16 GB, 32 GB for DS-MIX & DS-EX Pro(H)
- CPU: Intel® Core™ i7-8700
- GPU: NVIDIA GTX 1060 (6 GB RAM)
- USB: USB 3.0 port + additional USB port
- SSD: EXE location ≥ 2 GB, DentalOrder location ≥ 20 GB
- Power: AC input: 100-240 V, 1.5 A, 50-60 Hz
- Network: Internet connection required



INTRAORALSCAN MINIMUM SYSTEM REQUIREMENTS

System Requirements for using IntraoScan V3.0.6.4 or later

Item	Requirments
Operating System	Windows 10 Professional (64-bit) or later
Memory (RAM)	16GB or above
Display card (Nvidia discrete graphic card only)	NVIDIA RTX 2060 ,3060 or higher (RTX 30 series not including RTX 3050)
CPU (Intel processor only)	Intel(R) Core(TM) i7-8700 or above
Hard Drive	256GB SSD or above
Available Space	<ul style="list-style-type: none">• Software installation location with a minimum available space of 8GB• Order storge location with a minimum available space of 20GB
Screen Resolution	1920 x 1080 or higher
I/O Ports	More than 2 type-A USB 3.0(or higher) ports
Power supply	AC input: 100-240V ~ 50/60Hz, 1.0A

Environment requirment

Work temperature: 10°C ~ 40°C



INTRAORALSCAN MINIMUM SYSTEM REQUIREMENTS

- OS: Windows 10 Professional (64-bit), Windows 11
- RAM: 16 GB
- CPU: Intel® Core™ i7-8700
- GPU: NVIDIA RTX 3060, not including RTX 3050
- USB: ≥2 USB 3.0 port (type A)
- SSD: Total ≥256 GB, EXE location ≥8 GB, DentalOrder location ≥20 GB
- Resolution: ≥1920 x 1080
- Power: AC input: 100-240 V, 1.0 A, 50-60 Hz
- Network: Internet connection required



ACCUWARE MINIMUM SYSTEM REQUIREMENTS

System Requirements for using AccuWare V3.1.7.31

Item	Minimum Requirments	Recommended
Operating System	Windows 10, Windows 11	Windows 10, Windows 11
Memory (RAM)	8GB	16GB or above
Display card (Nvidia discrete graphics card)	NVIDIA GTX 750Ti	NVIDIA GTX 1050 Ti and above
CPU	Intel Xeon Processor E3-1230 3.30(8M Cache, 3.20 Ghz)	Intel Core i5-8500 Processor (9M Cache, up to 4.10 Ghz) or above
Power supply	AC input: 100-240V / 360W	

Environment Requirments for using AccuFab L4D, D1S printer

Item	Requirments	Recommended
Temperature	20°C - 35°C	23°C ± 2°C
Humidity	30% - 70%	50% ± 10%
Workspace	A neat, stable and level platform	



ACCUWARE MINIMUM SYSTEM REQUIREMENTS

AccuWare

- OS: Windows 10 Professional (64-bit), Windows 11
- RAM: 16 GB
- CPU: **Intel®** Core™ i5-8500 (9M Cache, up to 4.10 GHz)
- GPU: **NVIDIA** GTX 1050 Ti
- SSD: C:\\ Drive **after installation** ≥10 GB
- Power: AC input: 100-240 V, 360 W



ACCUWARE MINIMUM SYSTEM REQUIREMENTS

AccuFab L4D & D1S

- Temperature: $23^{\circ}\text{C} \pm 2^{\circ}\text{C}$
- Humidity: $50\% \pm 10\%$
- Workspace: A neat, stable, and level platform



STEP 2: DISABLE FIREWALL & UNINSTALL ANTIVIRUS SOFTWARE

Firewall


- Disabling Windows firewalls prevents SHINING 3D's software from being blocked during the download and installation processes

Antivirus


- Disabling antivirus software (McAfee, Norton, Avira, etc.) prevents SHINING 3D's software from being blocked during the download and installation processes

DISABLE FIREWALL


OPEN WINDOWS START MENU

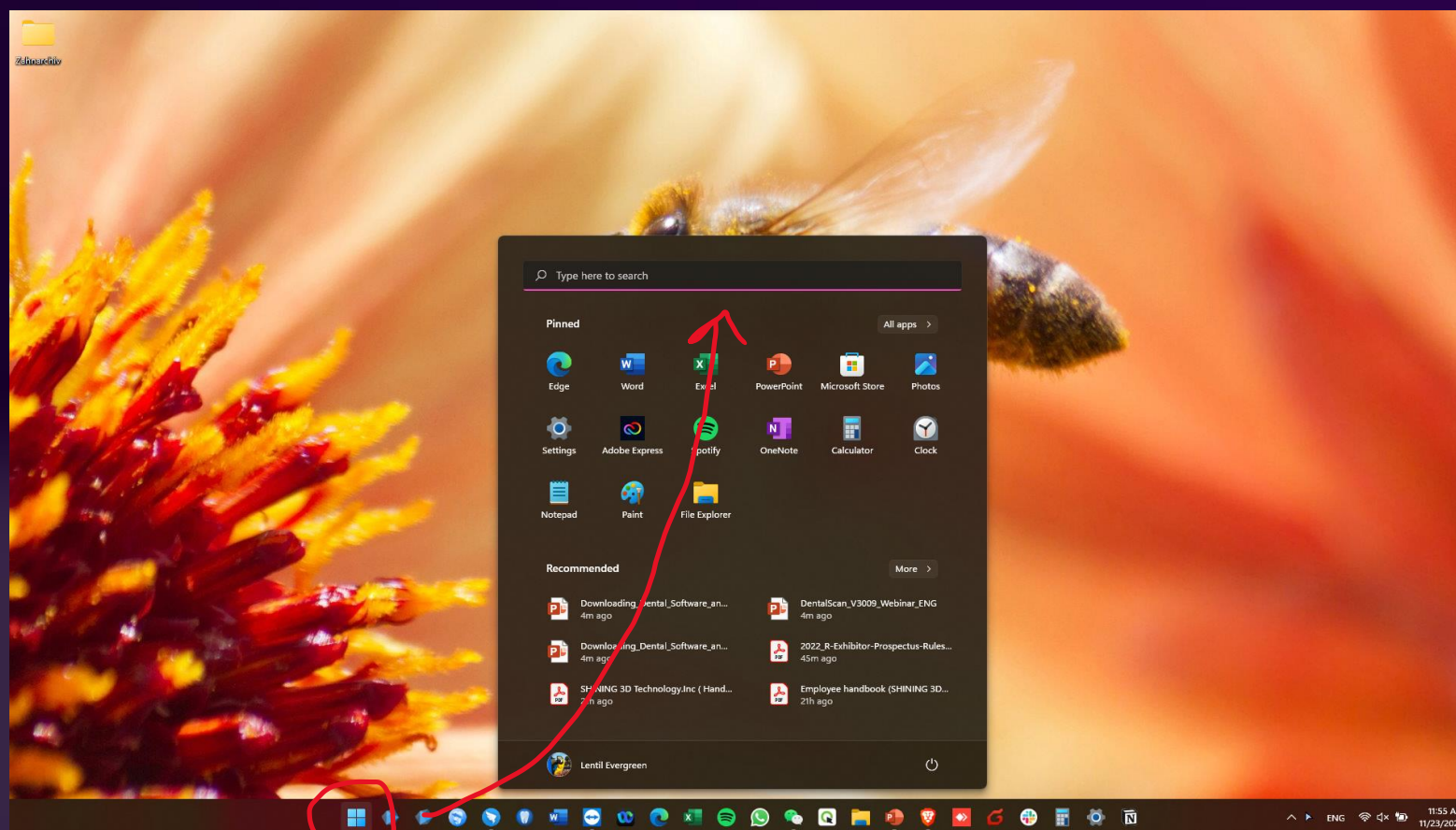


LOCATE WINDOWS FIREWALL SETTINGS

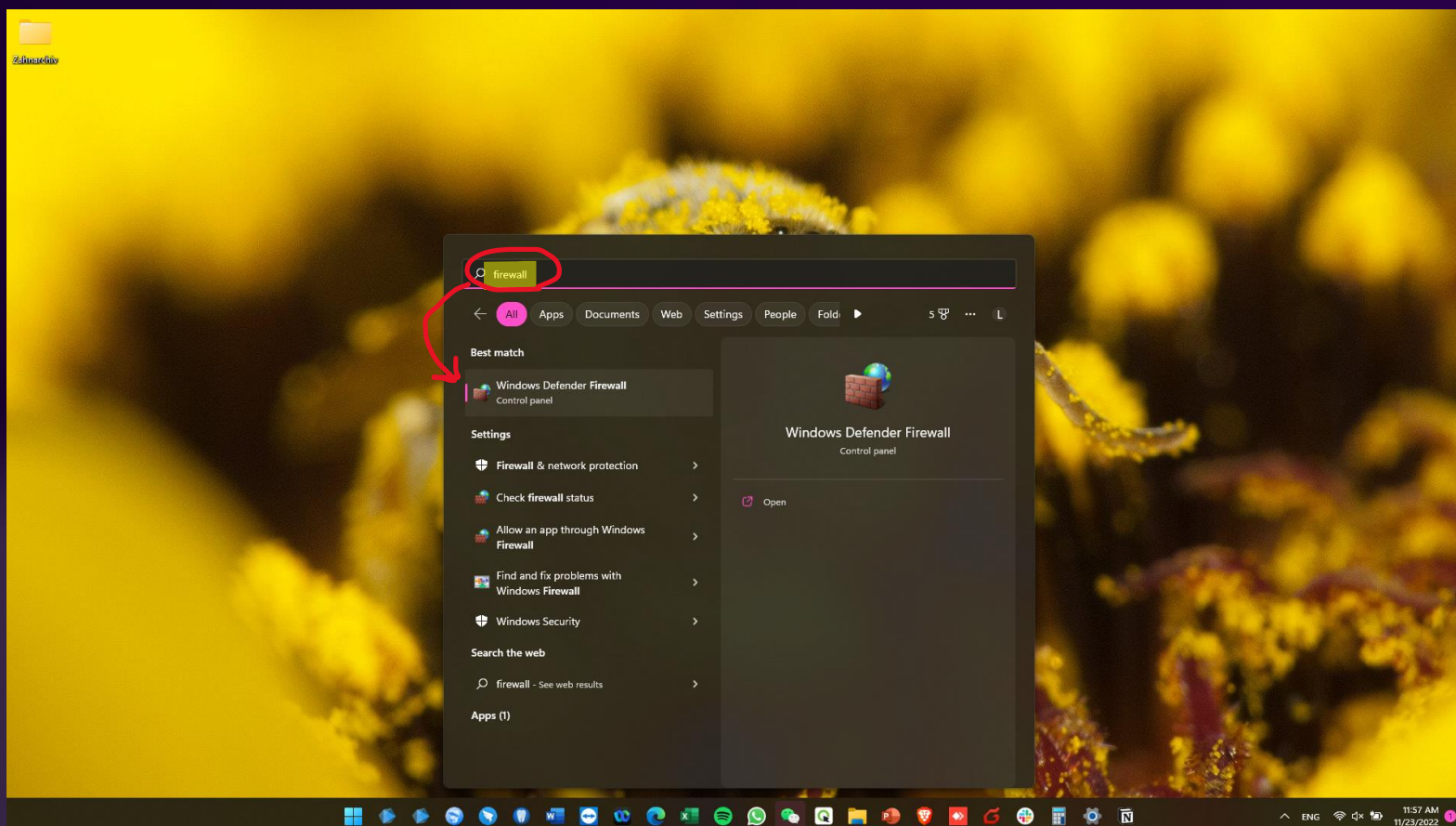


NAVIGATE TO “TURN WINDOWS DEFENDER FIREWALL ON OR OFF”

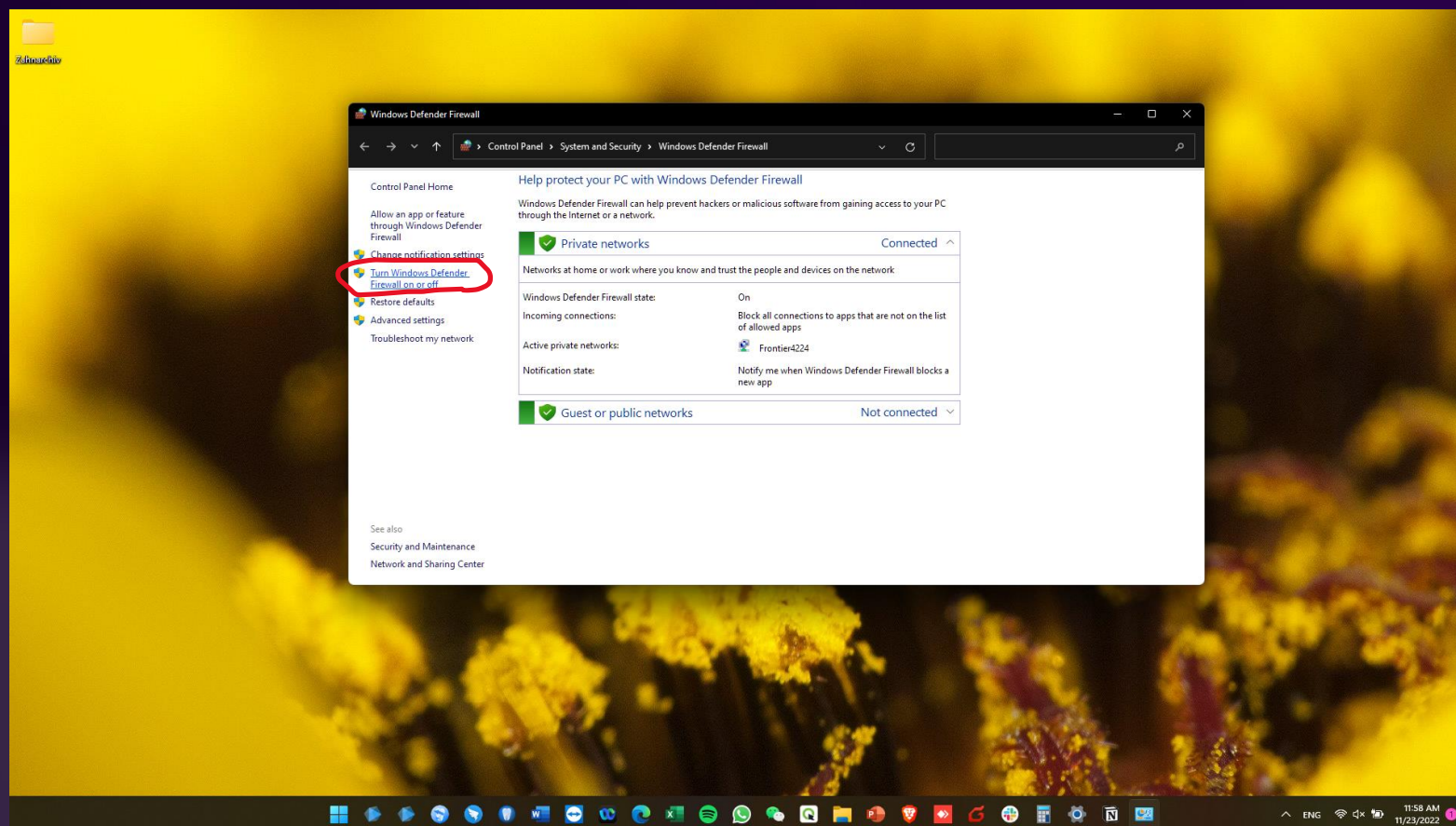


DISABLE PRIVATE & PUBLIC FIREWALLS

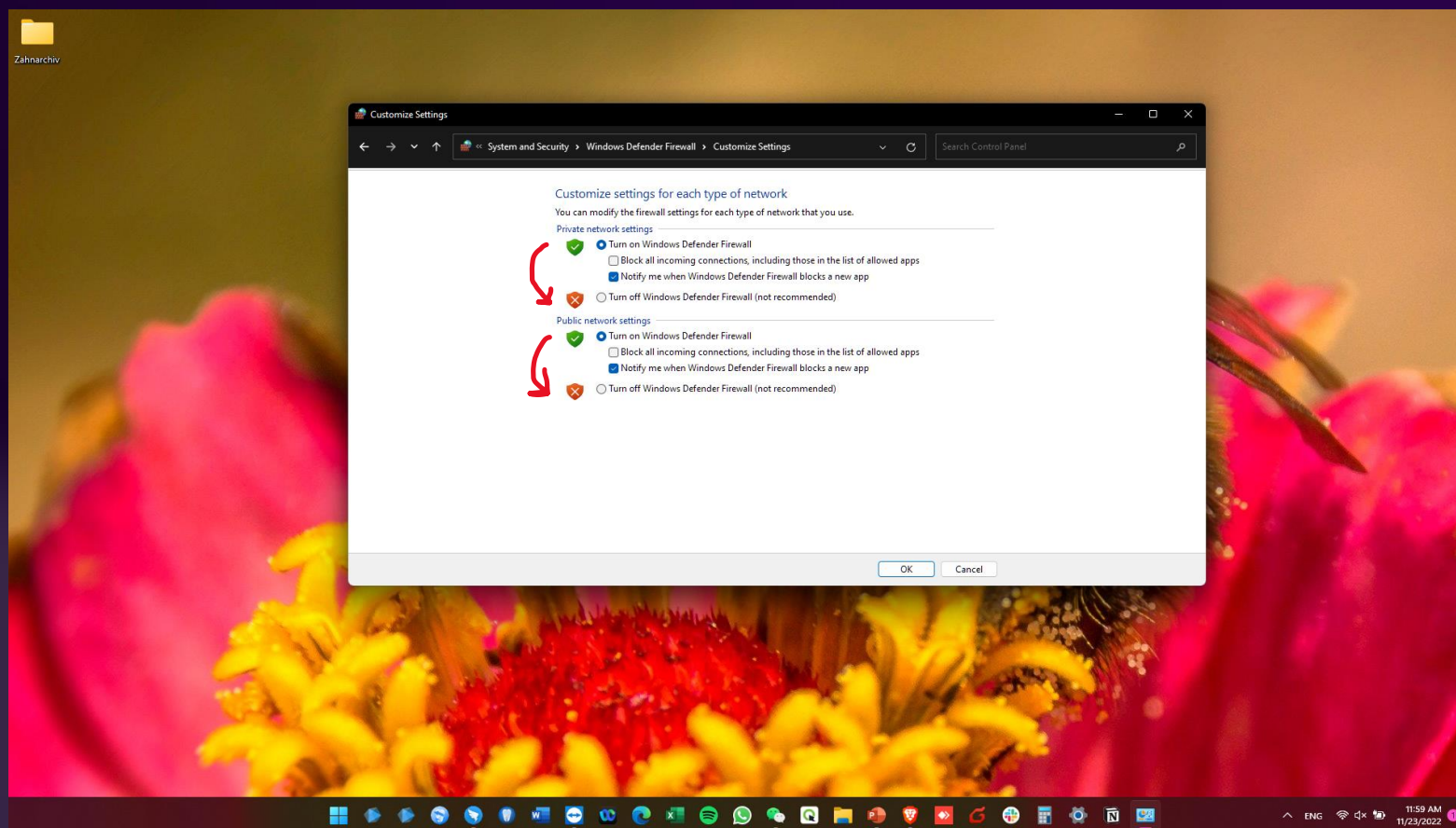

ONCE COMPLETE, THE SETTINGS SHOULD APPEAR AS FOLLOWS:

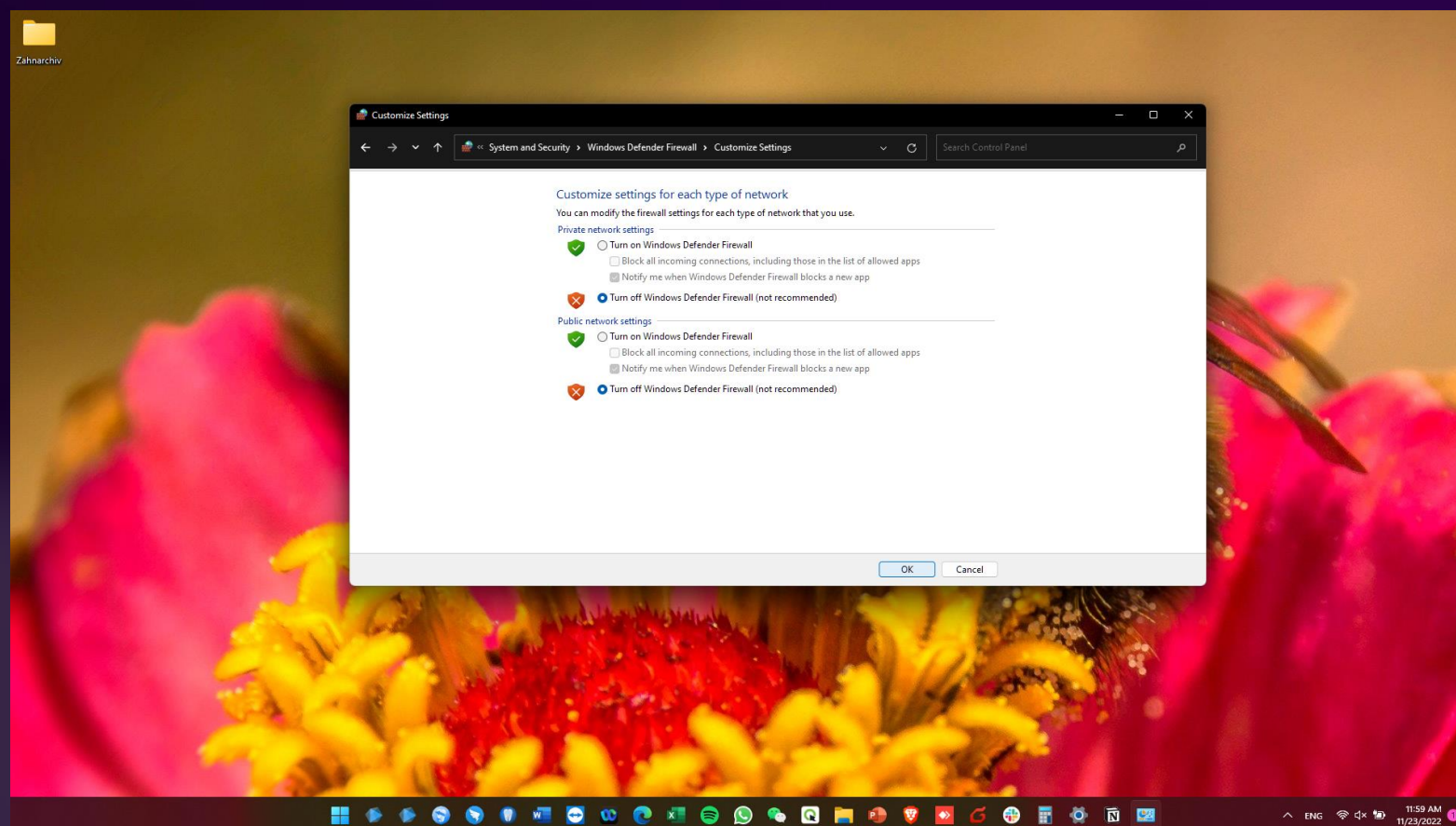


ONCE COMPLETE, THE SETTINGS SHOULD APPEAR AS FOLLOWS:

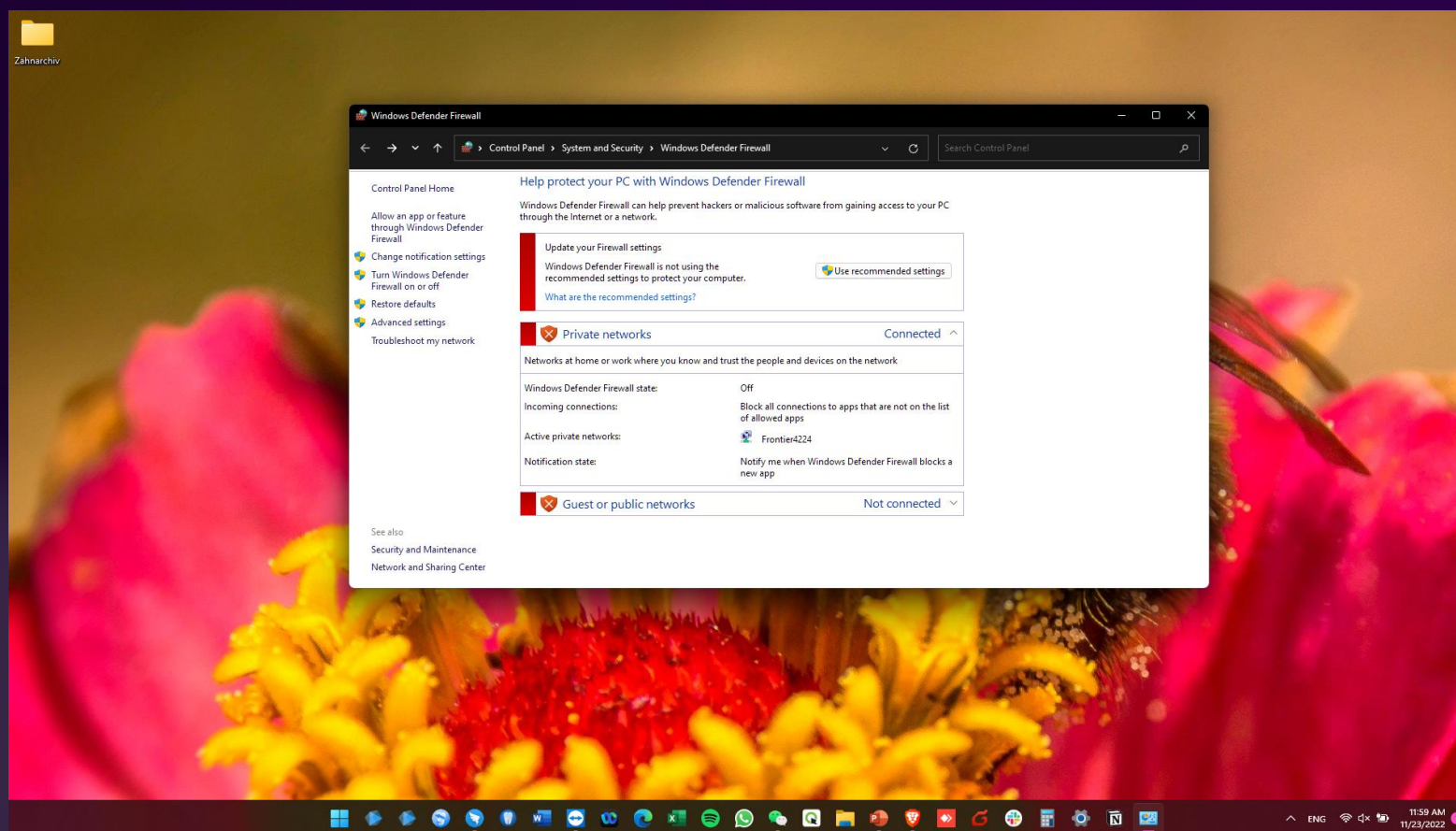


UNINSTALL ANTIVIRUS SOFTWARE


OPEN WINDOWS START MENU

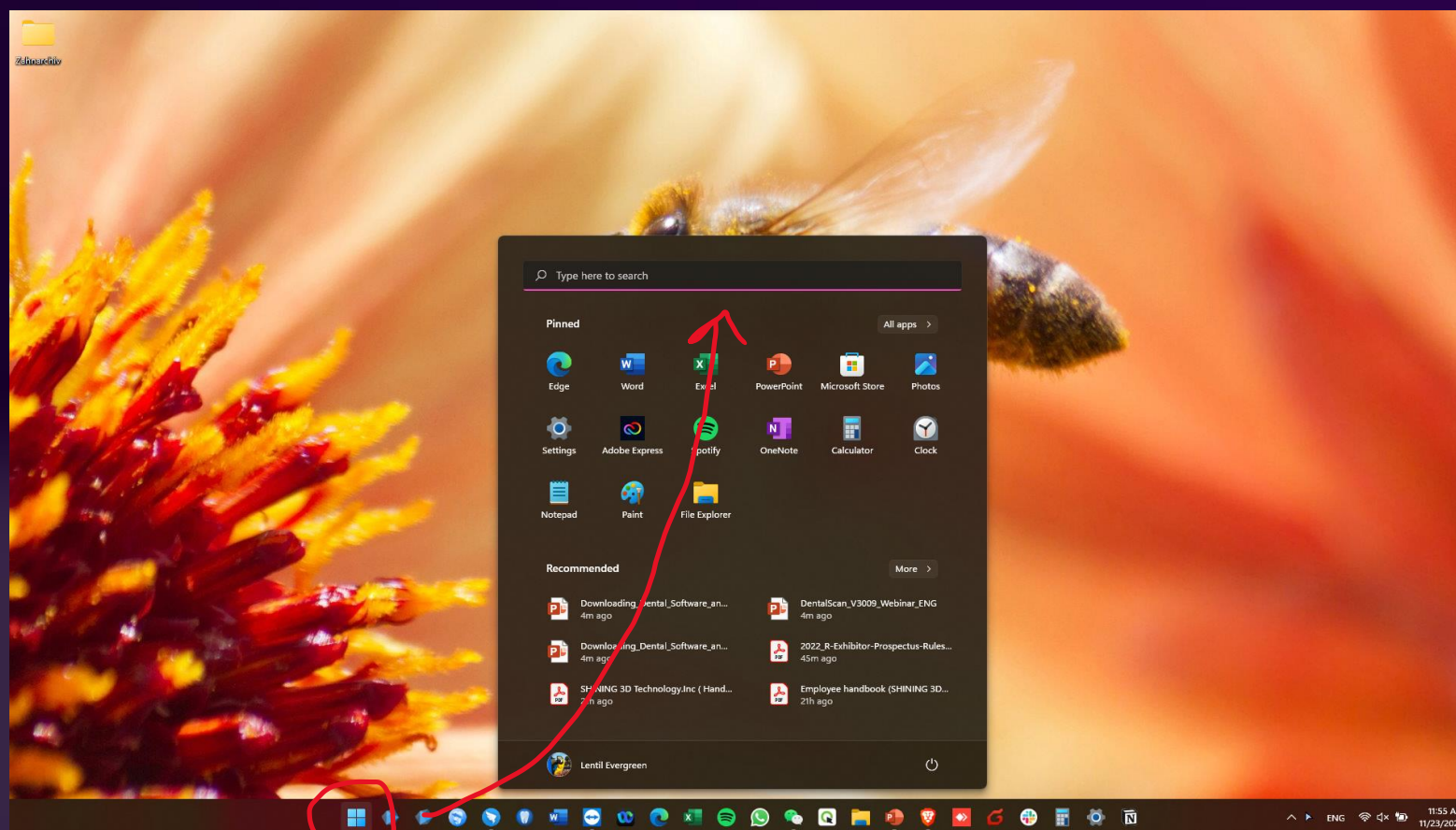

LOCATE “ADD OR REMOVE PROGRAMS”

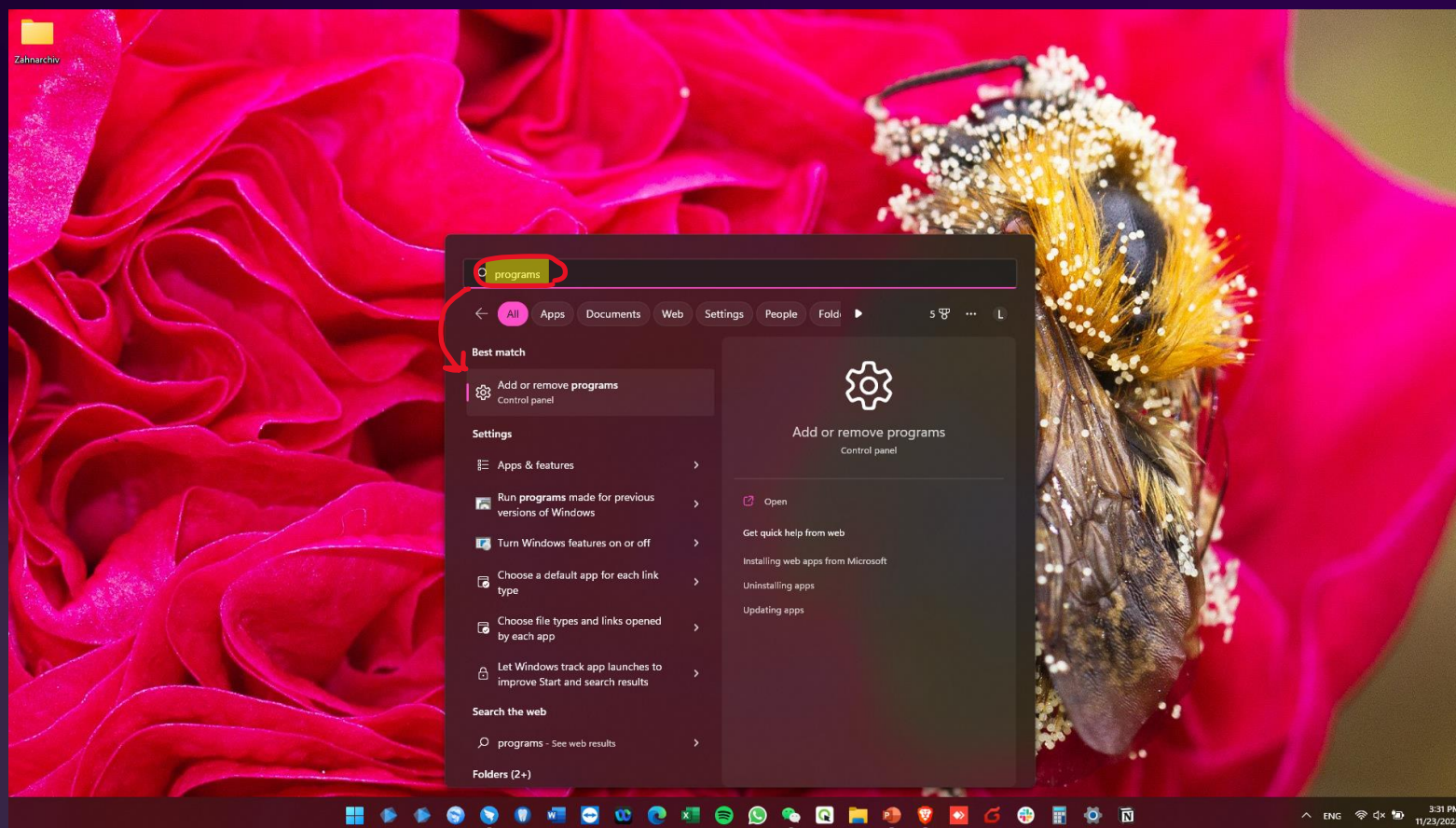

ON THIS PAGE, IDENTIFY ANY ANTIVIRUS SOFTWARE & UNINSTALL IT

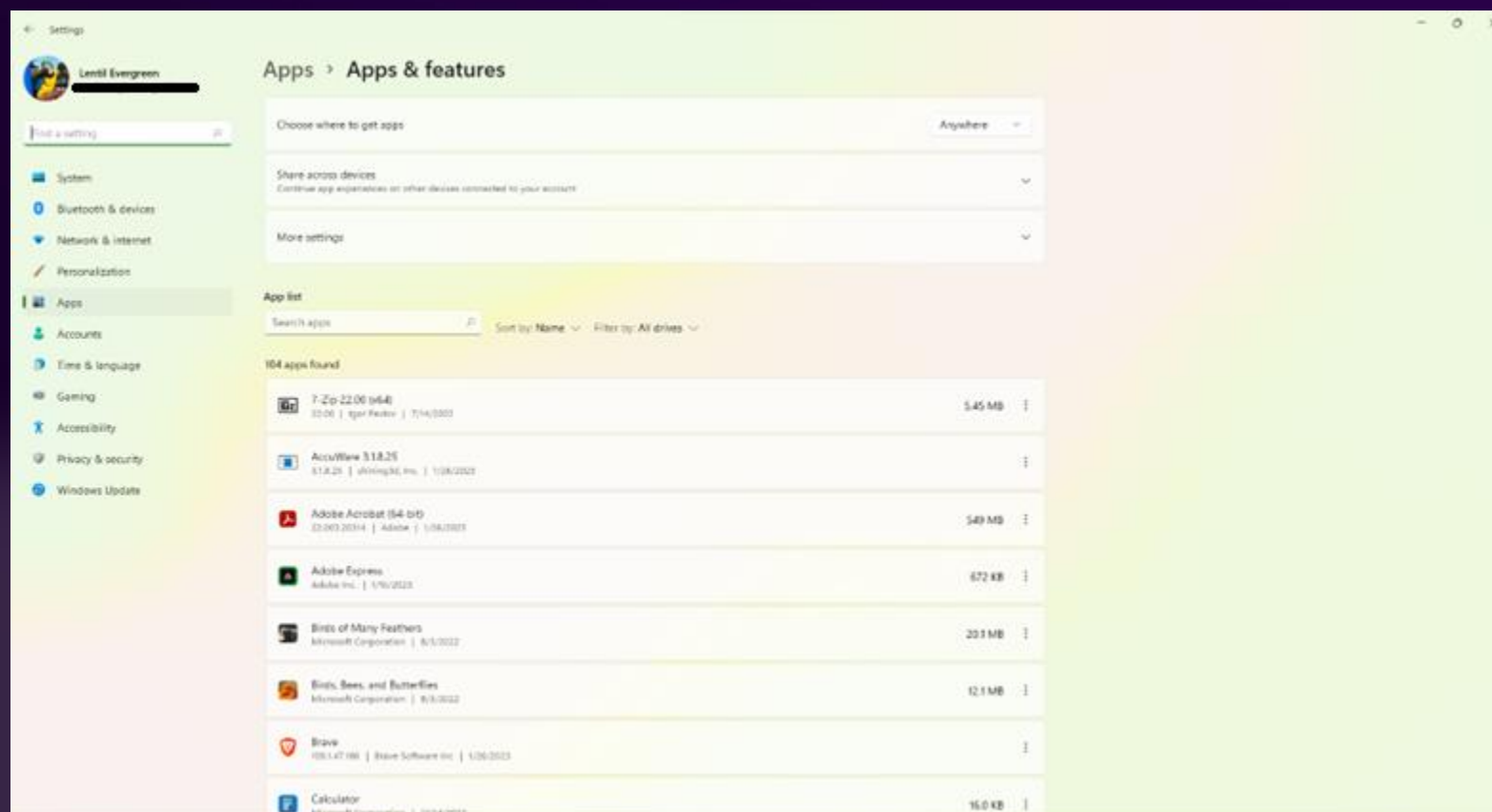

PLEASE BE SURE TO CHECK FOR ANY & ALL ANTIVIRUS BRANDS



**AS WELL, PLEASE ENSURE THAT THE ANTIVIRUS SOFTWARE
UNINSTALLS *COMPLETELY***



STEP 3: ENSURE GPU & WINDOWS OS DRIVERS ARE UP TO DATE

GPU Drivers


- Graphics cards (GPUs) need periodic updates in order to function properly, even if the GPU already meets minimum requirements

Windows OS


- Likewise, Windows OS can sometimes create software dysfunction if Windows updates have not been completed

CHECK & UPDATE GPU DRIVERS


OPEN WINDOWS START MENU

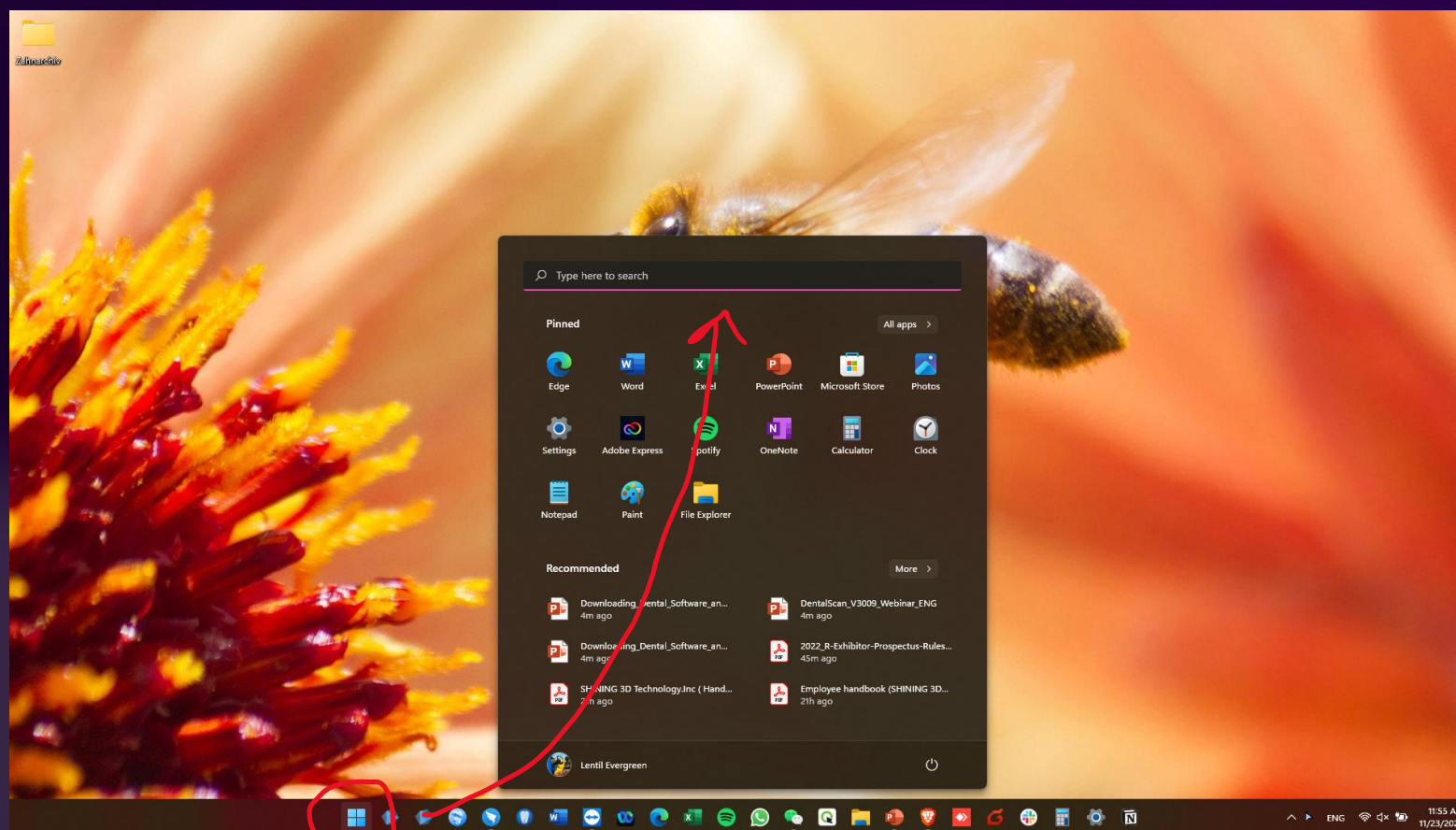


LOCATE DEVICE MANAGER

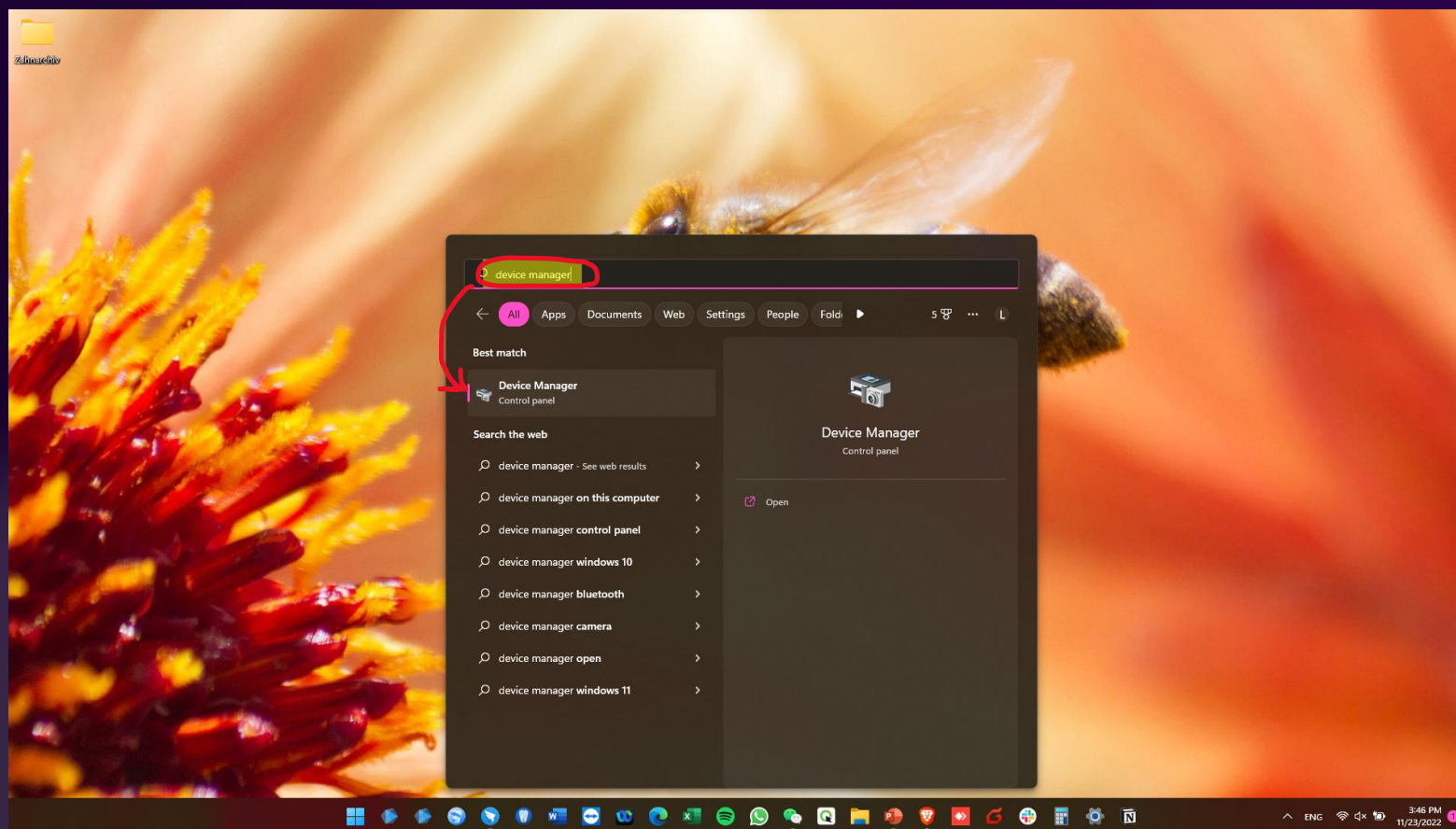


IDENTIFY NVIDIA GRAPHICS CARD UNDER “DISPLAY ADAPTERS”

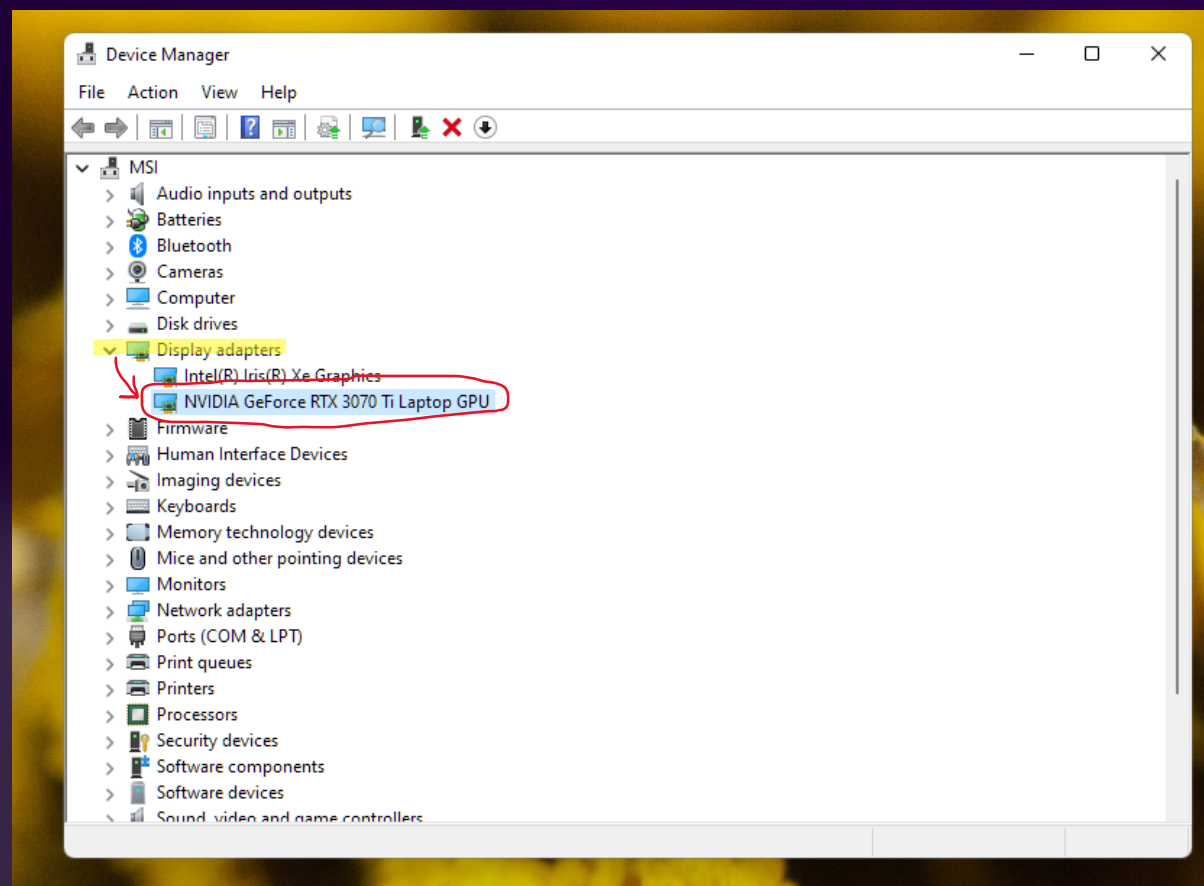


OPEN DEVICE PROPERTIES

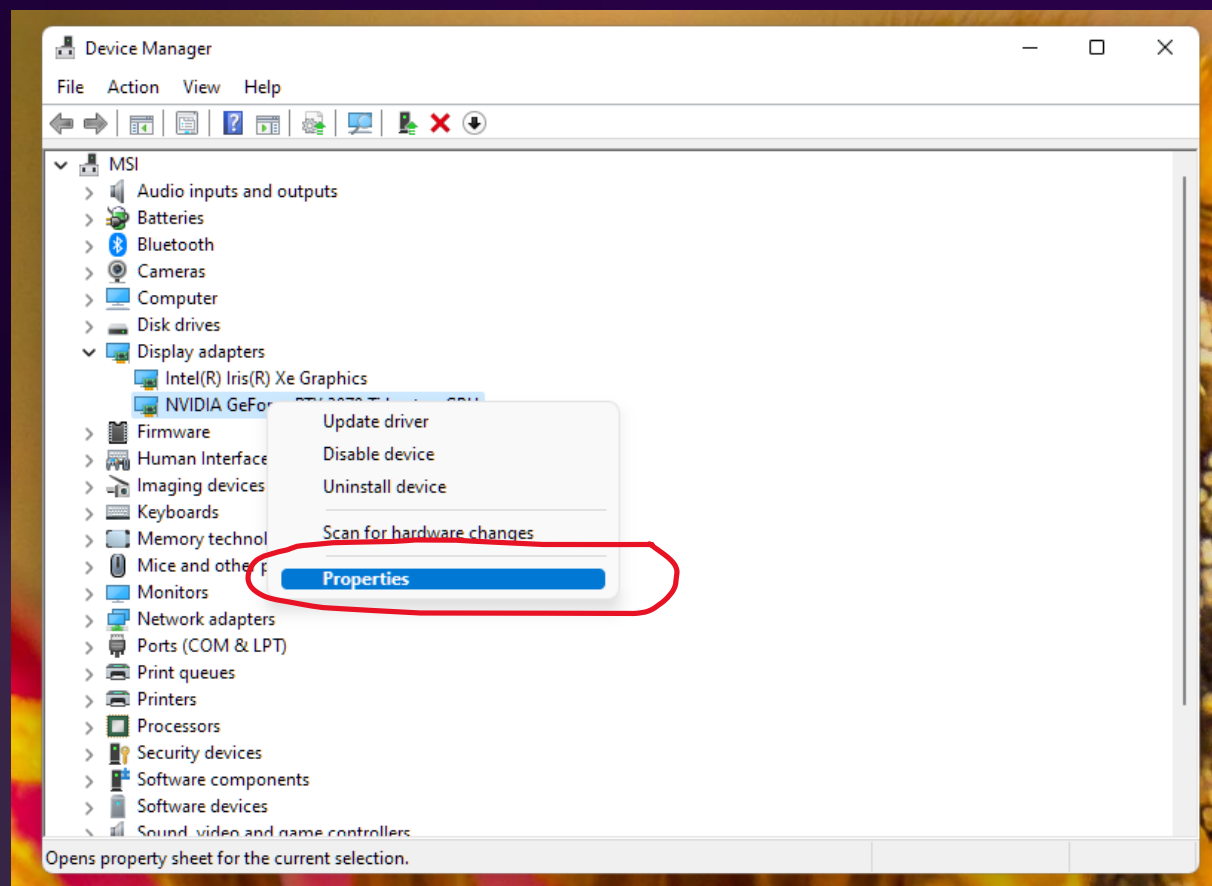

CHECK FOR DRIVER DATE IN DEVICE PROPERTIES

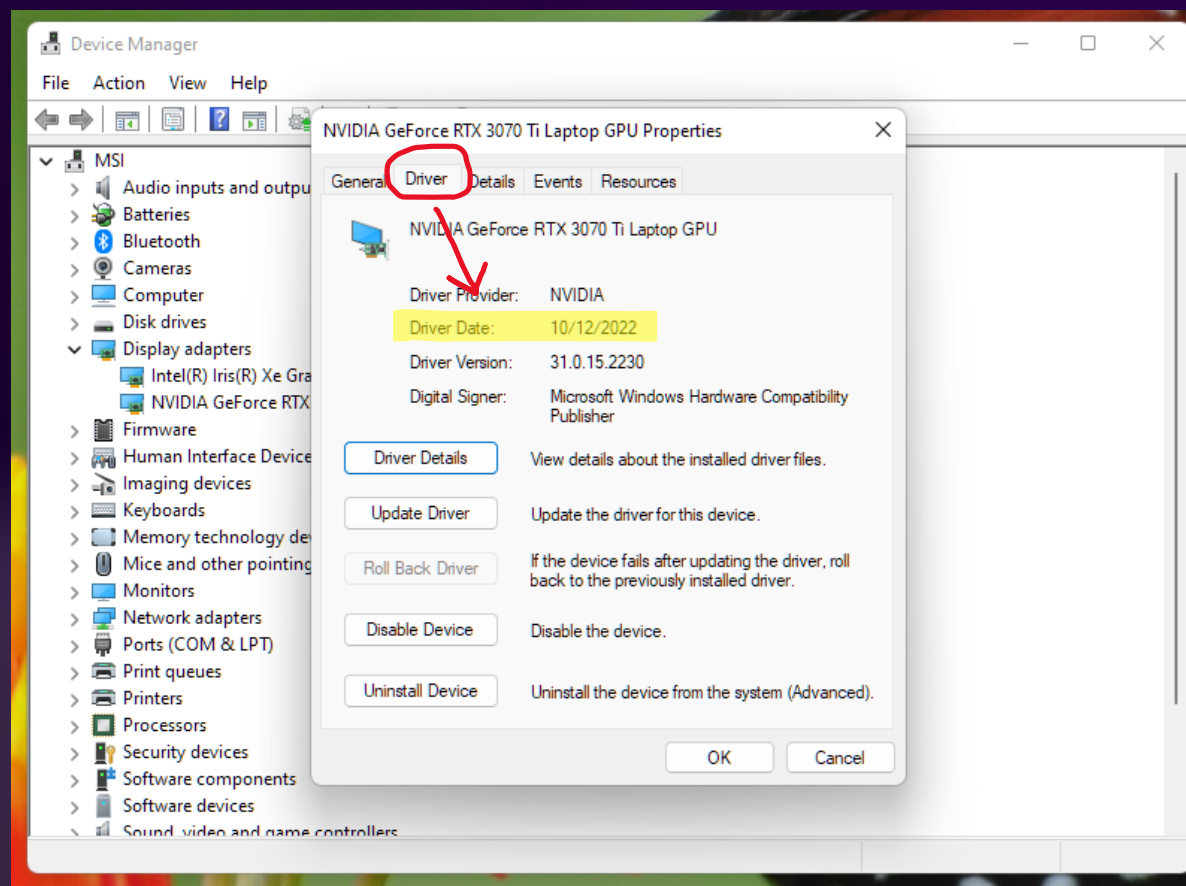

IF OUTDATED, SELECT GPU & DOWNLOAD LATEST DRIVER FROM NVIDIA'S WEBSITE

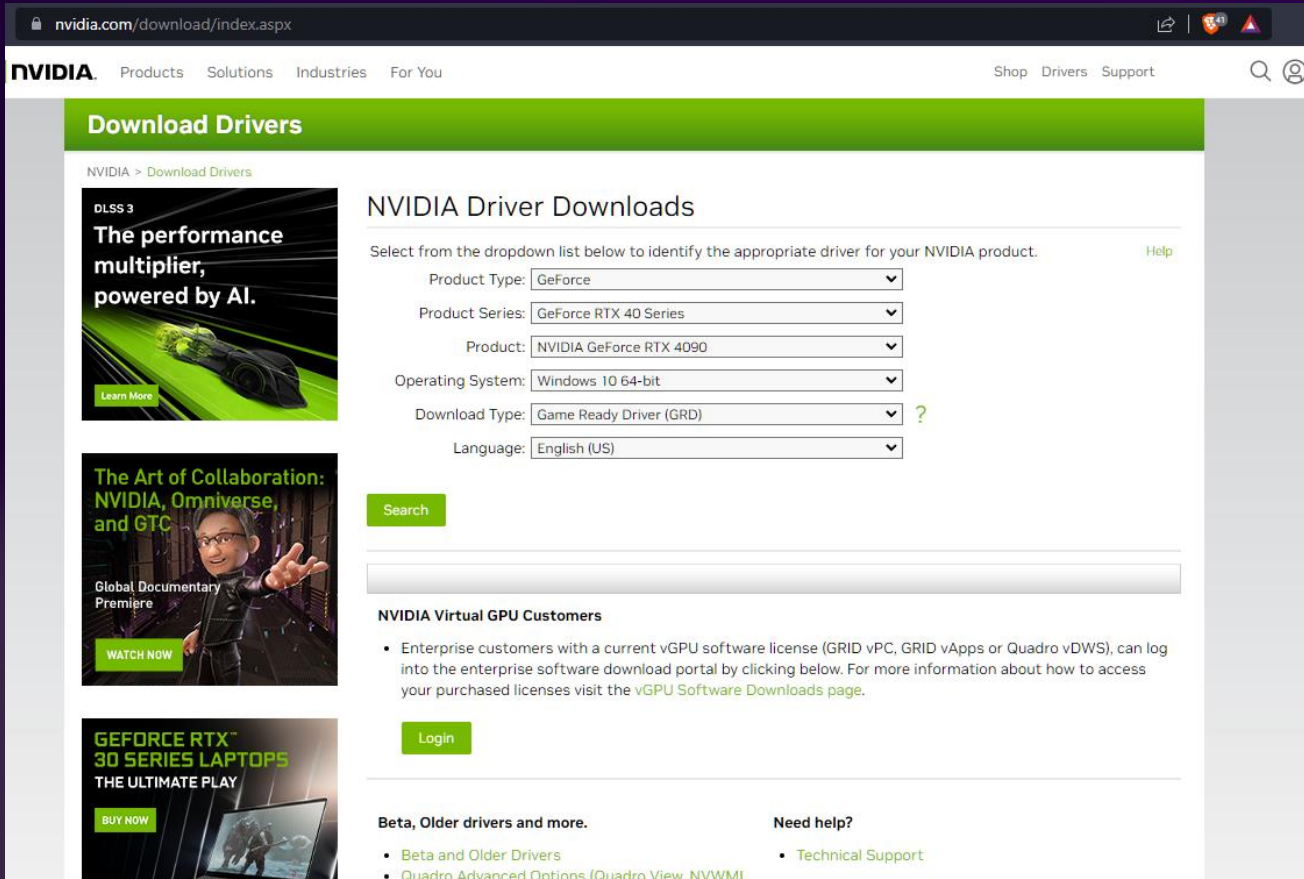

The screenshot shows the NVIDIA website's "Download Drivers" page. The page has a green header with the NVIDIA logo and navigation links. The main content area is titled "Download Drivers" and features a sidebar with promotional banners for DLSS 3, "The Art of Collaboration: NVIDIA, Omniverse, and GTC", and GeForce RTX 30 Series Laptops. The main section, "NVIDIA Driver Downloads", contains a form with dropdown menus for Product Type (GeForce), Product Series (GeForce RTX 40 Series), Product (NVIDIA GeForce RTX 4090), Operating System (Windows 10 64-bit), Download Type (Game Ready Driver (GRD)), and Language (English (US)). A "Search" button is located below the form. Below the form, there is a section for "NVIDIA Virtual GPU Customers" with a "Login" button and a list of links for "Beta, Older drivers and more." and "Need help?".

Download Drivers

NVIDIA > Download Drivers

NVIDIA Driver Downloads

Select from the dropdown list below to identify the appropriate driver for your NVIDIA product. [Help](#)

Product Type:

Product Series:

Product:

Operating System:

Download Type: ?

Language:

[Search](#)

NVIDIA Virtual GPU Customers

- Enterprise customers with a current vGPU software license (GRID vPC, GRID vApps or Quadro vDWS), can log into the enterprise software download portal by clicking below. For more information about how to access your purchased licenses visit the [vGPU Software Downloads page](#).

[Login](#)


Beta, Older drivers and more.

- [Beta and Older Drivers](#)
- [Quadro Advanced Options \(Quadro View, NVVM, ...\)](#)

Need help?

- [Technical Support](#)

IF OUTDATED, SELECT GPU & DOWNLOAD LATEST DRIVER FROM NVIDIA'S WEBSITE

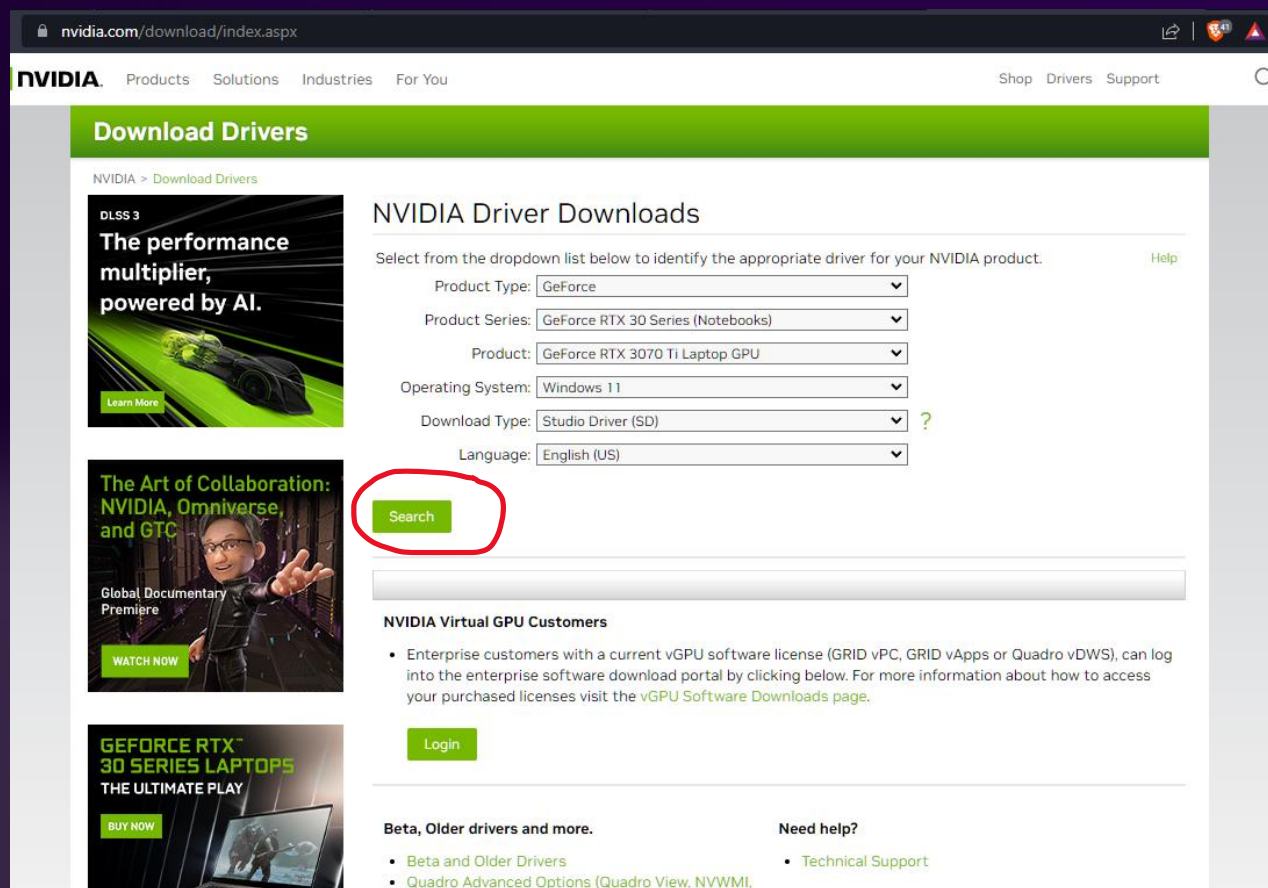

The screenshot shows the NVIDIA website's driver download page. The URL in the browser is `nvidia.com/download/index.aspx`. The page has a green header with the NVIDIA logo and navigation links: Products, Solutions, Industries, For You, Shop, Drivers, Support, and a search icon. Below the header is a green bar with the text "Download Drivers". The main content area is titled "NVIDIA Driver Downloads" and includes a "Help" link. It features a selection form with dropdown menus for Product Type (GeForce), Product Series (GeForce RTX 30 Series (Notebooks)), Product (GeForce RTX 3070 Ti Laptop GPU), Operating System (Windows 11), Download Type (Studio Driver (SD)), and Language (English (US)). A red circle highlights the "Search" button. Below the form is a "Login" button. The page also includes promotional banners for DLSS 3, "The Art of Collaboration: NVIDIA, Omniverse, and GTC", and GeForce RTX 30 Series Laptops. At the bottom, there are links for "Beta, Older drivers and more." and "Need help?" with sub-links for "Beta and Older Drivers", "Technical Support", and "Quadro Advanced Options (Quadro View, NVWMI)".

Download Drivers

NVIDIA > Download Drivers

NVIDIA Driver Downloads

Select from the dropdown list below to identify the appropriate driver for your NVIDIA product. [Help](#)

Product Type:

Product Series:

Product:

Operating System:

Download Type: ?

Language:

Search

NVIDIA Virtual GPU Customers

- Enterprise customers with a current vGPU software license (GRID vPC, GRID vApps or Quadro vDWS), can log into the enterprise software download portal by clicking below. For more information about how to access your purchased licenses visit the [vGPU Software Downloads page](#).

Login


Beta, Older drivers and more.

- [Beta and Older Drivers](#)
- [Quadro Advanced Options \(Quadro View, NVWMI\)](#)

Need help?

- [Technical Support](#)

IF OUTDATED, SELECT GPU & DOWNLOAD LATEST DRIVER FROM NVIDIA'S WEBSITE

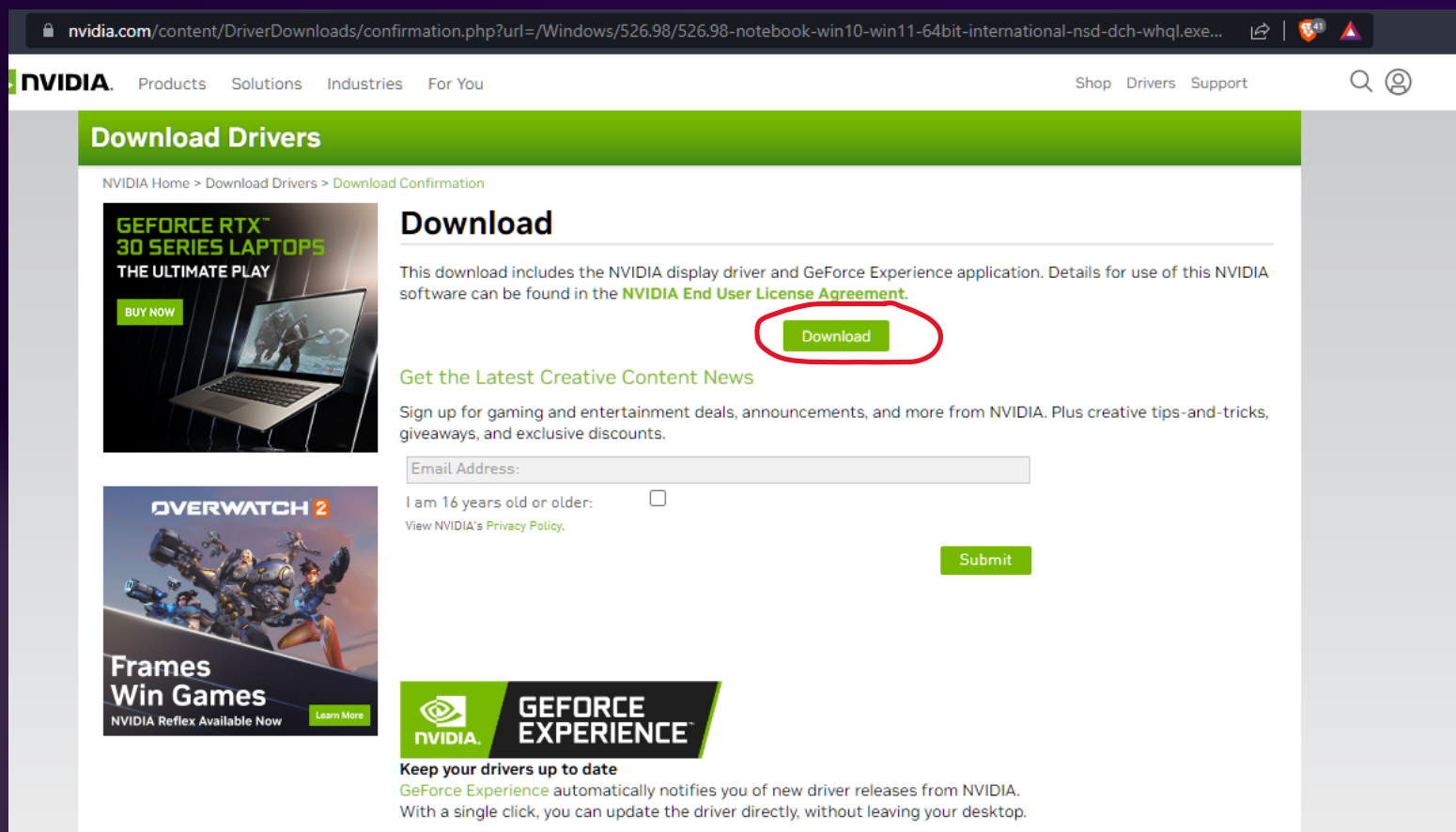

nvidia.com/content/DriverDownloads/confirmation.php?url=/Windows/526.98/526.98-notebook-win10-win11-64bit-international-nsd-dch-whql.exe...

NVIDIA Products Solutions Industries For You Shop Drivers Support

Download Drivers

NVIDIA Home > Download Drivers > Download Confirmation

Download

This download includes the NVIDIA display driver and GeForce Experience application. Details for use of this NVIDIA software can be found in the [NVIDIA End User License Agreement](#).

[Download](#)

Get the Latest Creative Content News

Sign up for gaming and entertainment deals, announcements, and more from NVIDIA. Plus creative tips-and-tricks, giveaways, and exclusive discounts.

Email Address:

I am 16 years old or older: ☐

[View NVIDIA's Privacy Policy](#)

[Submit](#)

GEFORCE EXPERIENCE


Keep your drivers up to date

GeForce Experience automatically notifies you of new driver releases from NVIDIA. With a single click, you can update the driver directly, without leaving your desktop.


GEFORCE RTX™ 30 SERIES LAPTOPS
THE ULTIMATE PLAY
[BUY NOW](#)

OVERWATCH 2
Frames Win Games
NVIDIA Reflex Available Now [Learn More](#)


RUN DRIVER UPDATE PACKAGE

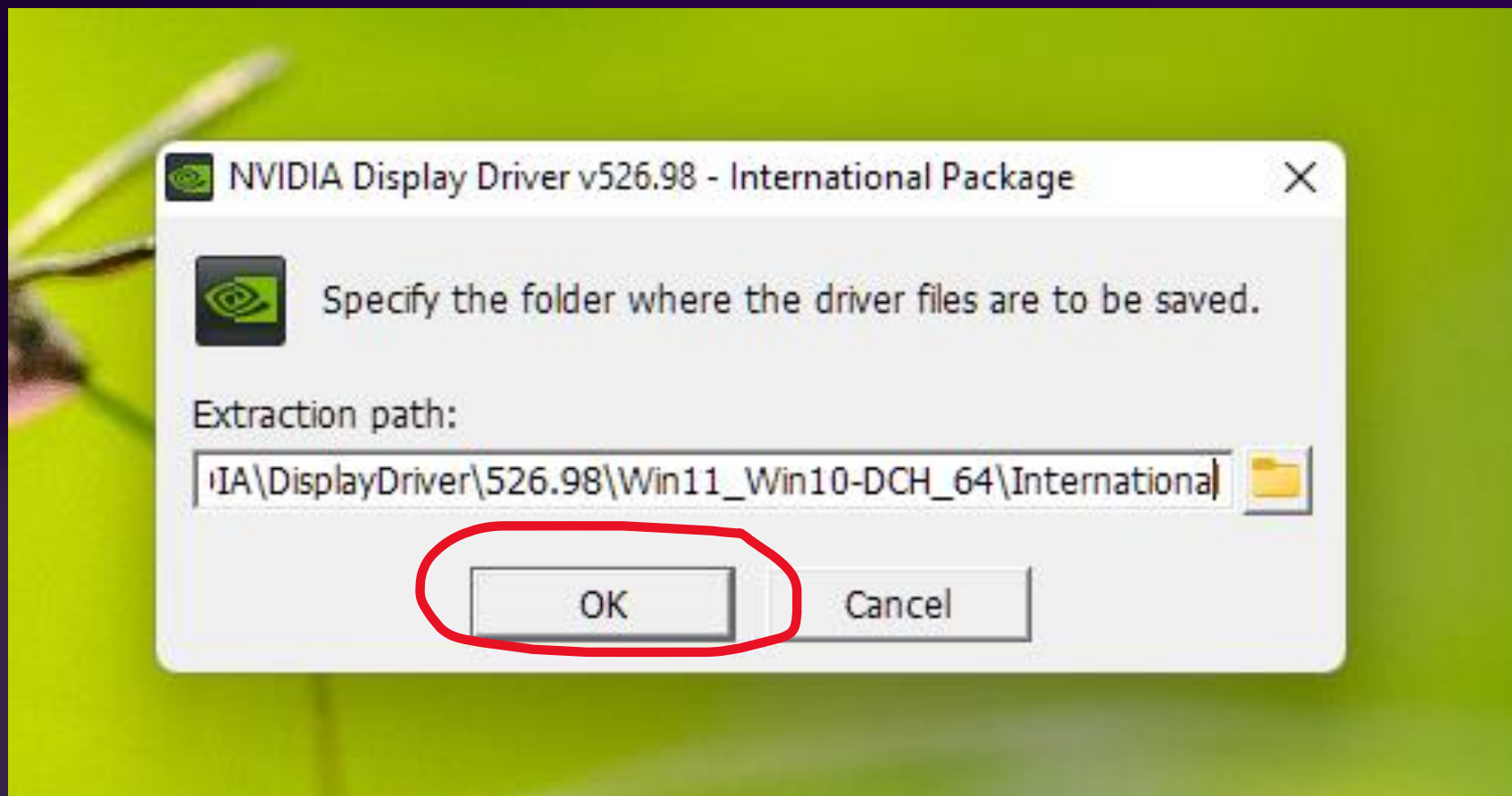


PLEASE BE SURE TO SELECT ONLY THE NVIDIA GRAPHICS DRIVER

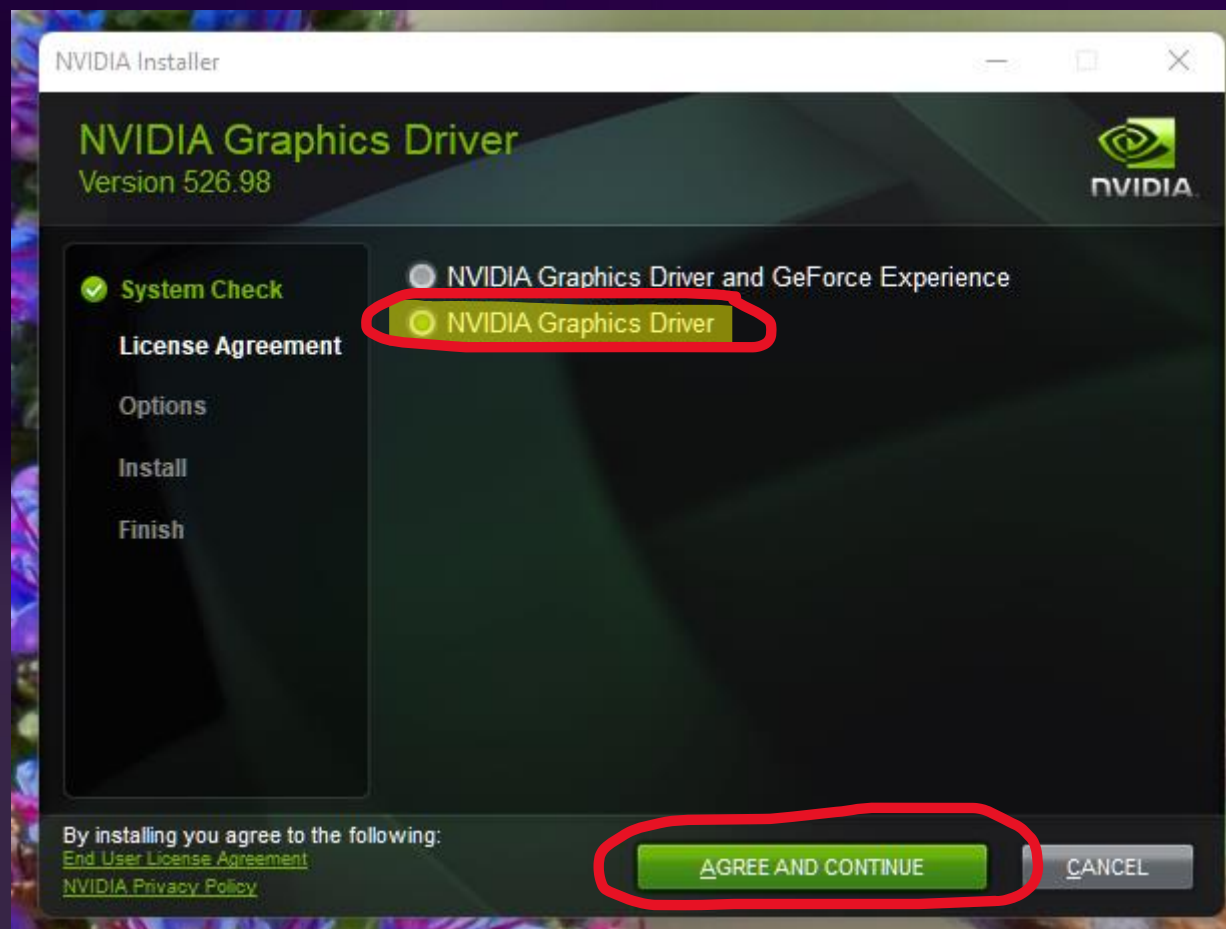

PLEASE BE SURE TO SELECT EXPRESS

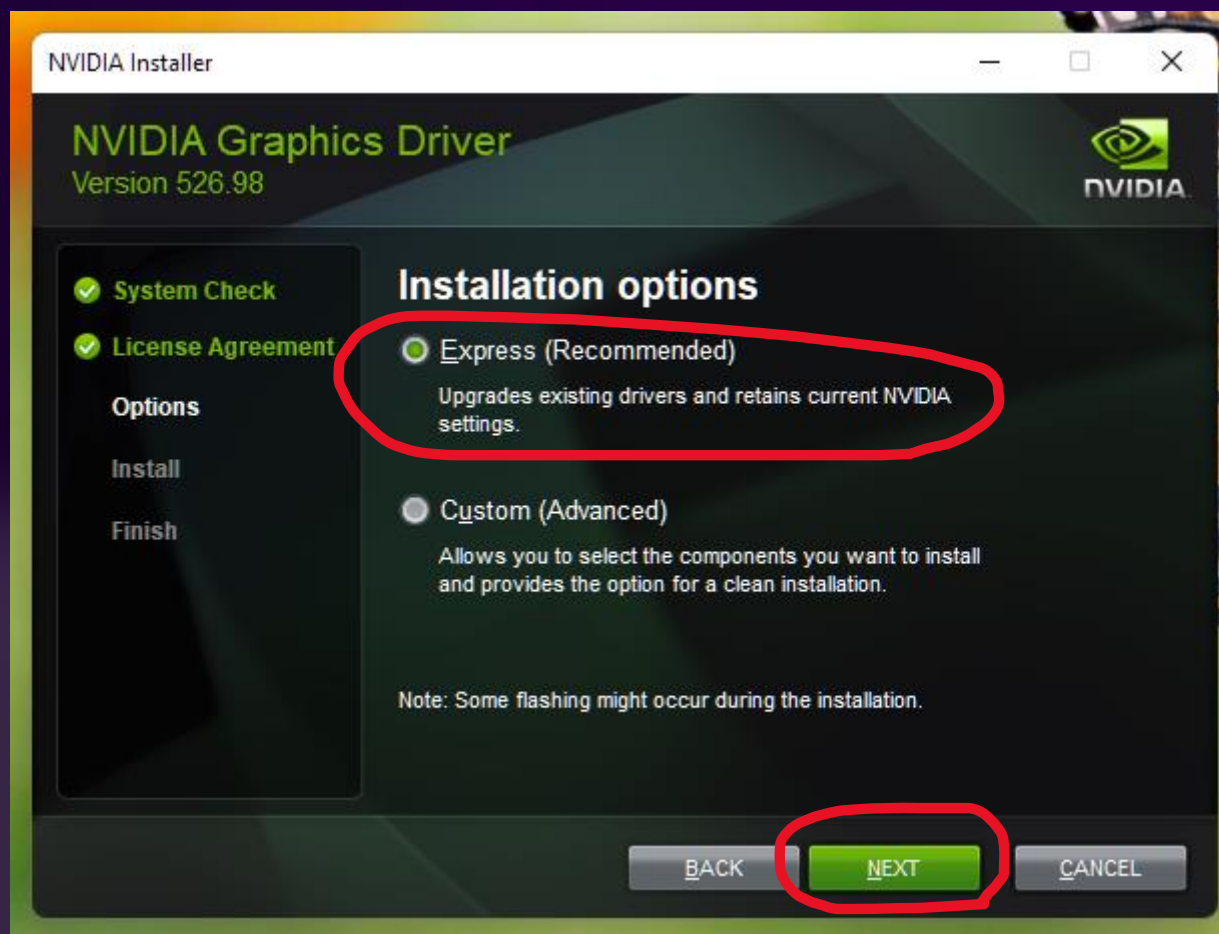


ONCE FINISHED, THE GPU DRIVER SHOULD BE UPDATED TO THE LATEST VERSION

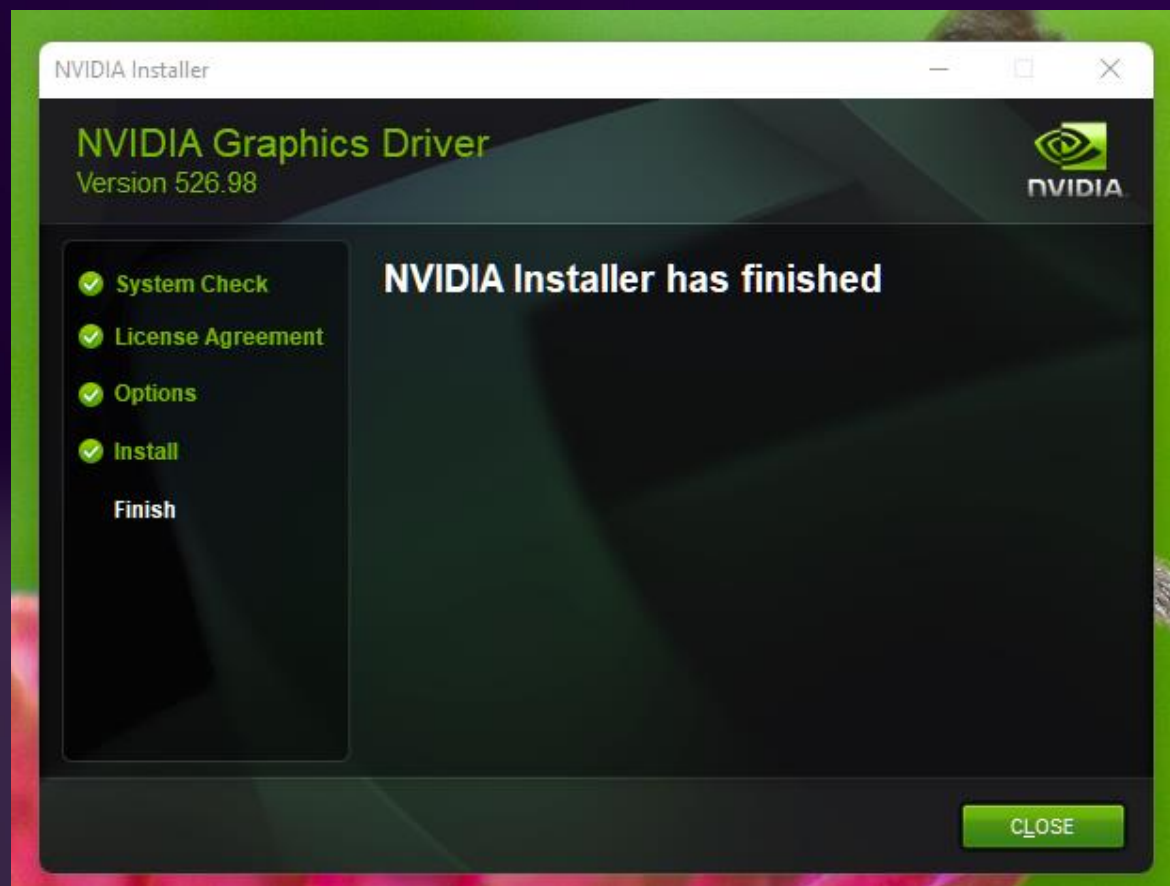


CHECK & UPDATE WINDOWS OS

OPEN THE WINDOWS START MENU

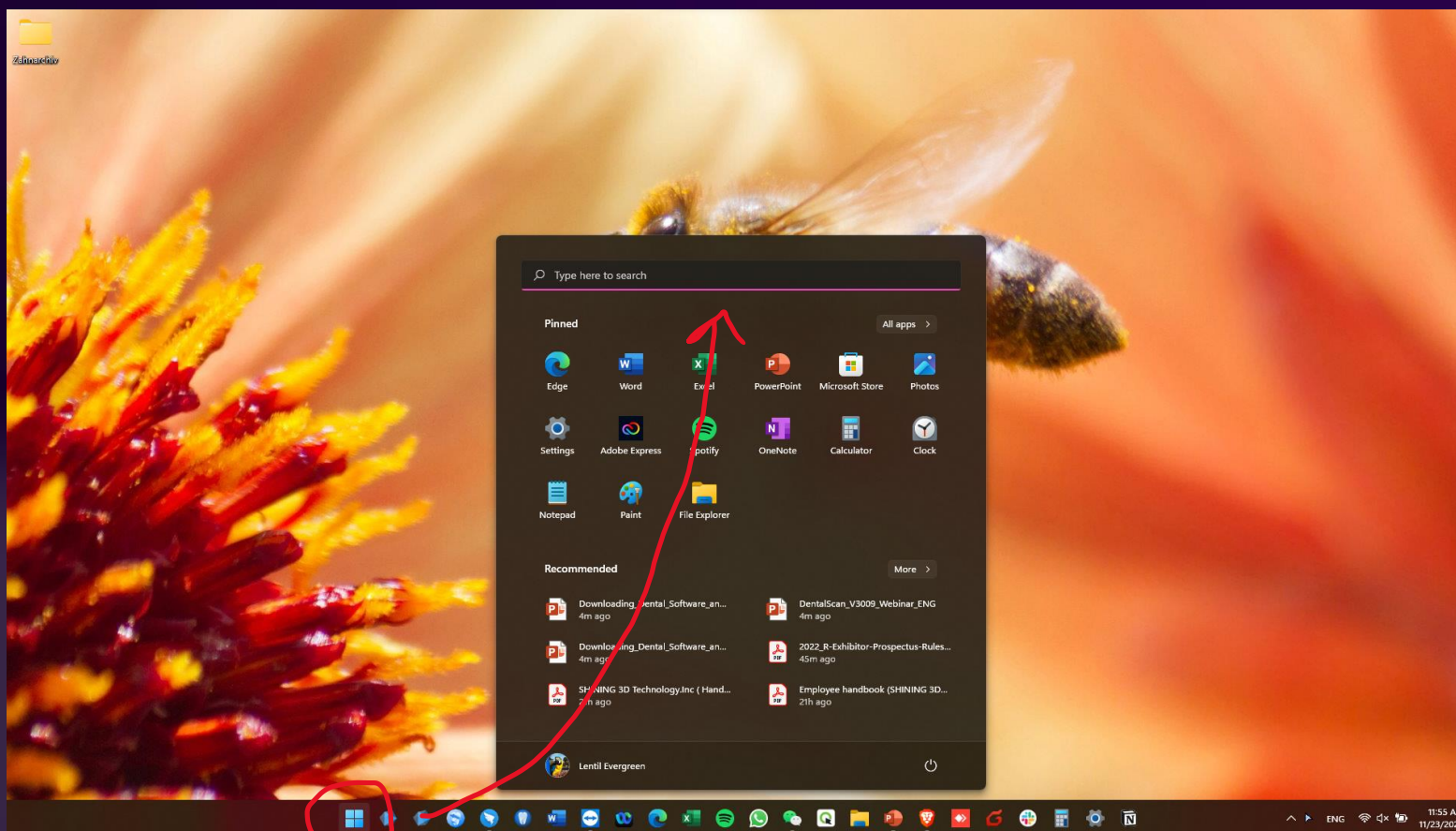


LOCATE WINDOWS UPDATES

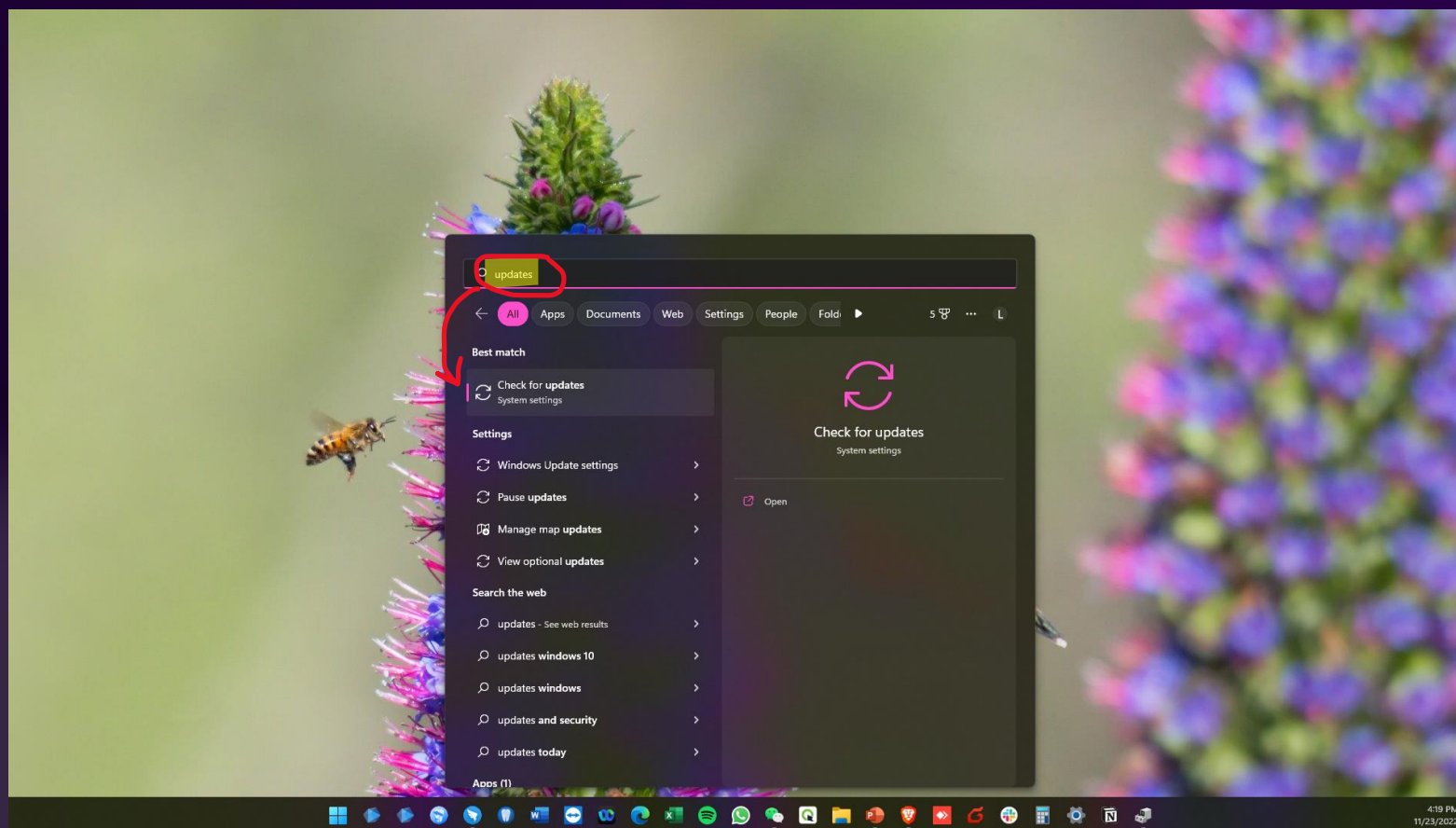


CHECK FOR WINDOWS UPDATES & INSTALL ALL


Windows Update




You're up to date
Last checked: Today, 9:18 AM


[Check for updates](#)


 2022-11 Cumulative Update for Windows 11 for x64-based Systems (KB5019157) is available.


[Download & install](#) 


More options

 Pause updates


[Pause for 1 week](#) 


 Update history




 Advanced options


Delivery optimization, **optional updates**, active hours, other update settings



 Windows Insider Program


Get preview builds of Windows to share feedback on new features and updates



 [Get help](#)


PLEASE BE SURE TO CHECK FOR OPTIONAL UPDATES

Windows Update



You're up to date
Last checked: Today, 9:18 AM


Check for updates

 2022-11 Cumulative Update for Windows 11 for x64-based Systems (KB5019157) is available.


Download & install

×


More options

 Pause updates

Pause for 1 week ▾


 Update history

>

 Advanced options


Delivery optimization, optional updates, active hours, other update settings

>


 Windows Insider Program

Get preview builds of Windows to share feedback on new features and updates

>


 Get help

AS WELL, IF AN UPDATE REQUIRES THE COMPUTER TO RESTART, RE-CHECK FOR NEW UPDATES

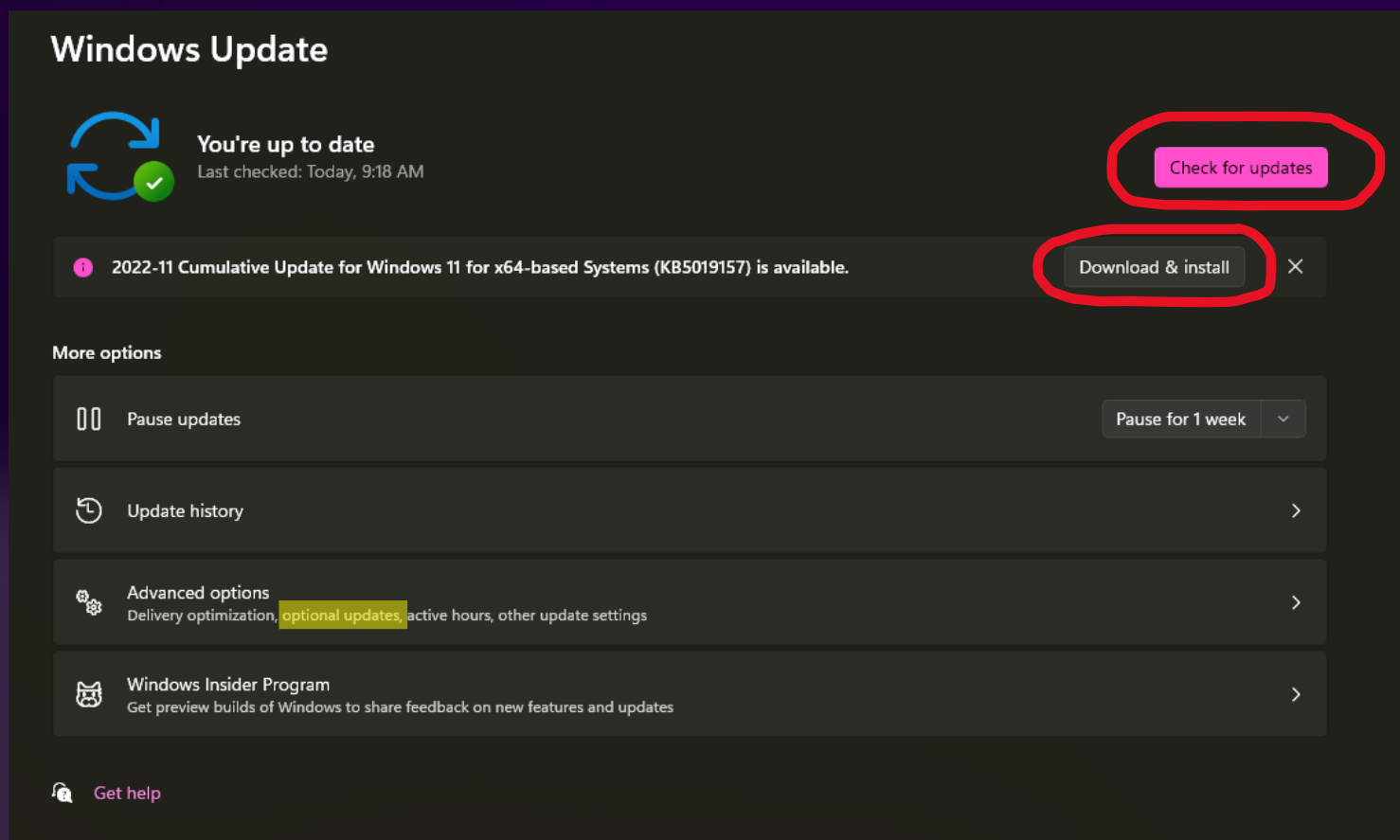



The screenshot shows the Windows Update application window. At the top, it says "Windows Update". Below that, there's a status section with a circular arrow icon and a green checkmark, stating "You're up to date" and "Last checked: Today, 9:18 AM". To the right of this status is a red-outlined button labeled "Check for updates". Below the status section, there's a notification bar with an information icon, stating "2022-11 Cumulative Update for Windows 11 for x64-based Systems (KB5019157) is available." To the right of this notification is a red-outlined button labeled "Download & install" and a close button (X). Below the notification bar, there's a section titled "More options" with four items: "Pause updates" (with a pause icon and a dropdown menu set to "Pause for 1 week"), "Update history" (with a clock icon and a right arrow), "Advanced options" (with a gear icon and a right arrow, and subtext "Delivery optimization, optional updates, active hours, other update settings"), and "Windows Insider Program" (with a cat icon and a right arrow, and subtext "Get preview builds of Windows to share feedback on new features and updates"). At the bottom left, there's a help icon and the text "Get help".

Windows Update


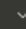

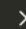
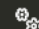
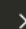


 You're up to date
Last checked: Today, 9:18 AM


[Check for updates](#)

 2022-11 Cumulative Update for Windows 11 for x64-based Systems (KB5019157) is available.

[Download & install](#) 

More options

-  Pause updates [Pause for 1 week](#) 
-  Update history 
-  Advanced options
Delivery optimization, **optional updates**, active hours, other update settings 
-  Windows Insider Program
Get preview builds of Windows to share feedback on new features and updates 

 [Get help](#)

STEP 4: DOWNLOAD 7ZIP & DENTAL SOFTWARE

7zip


- An unpacking software that allows for the dental software to be unzipped after downloading, which allows for a faster download process. Please *do not* use WinRAR, Windows extractor, etc.

Dental Software

- The latest versions of all SHINING 3D's dental software can be found at Shining3DDental.com under the Support Center section

DOWNLOAD & INSTALL 7ZIP

IN A BROWSER WINDOW, NAVIGATE TO 7ZIP.ORG

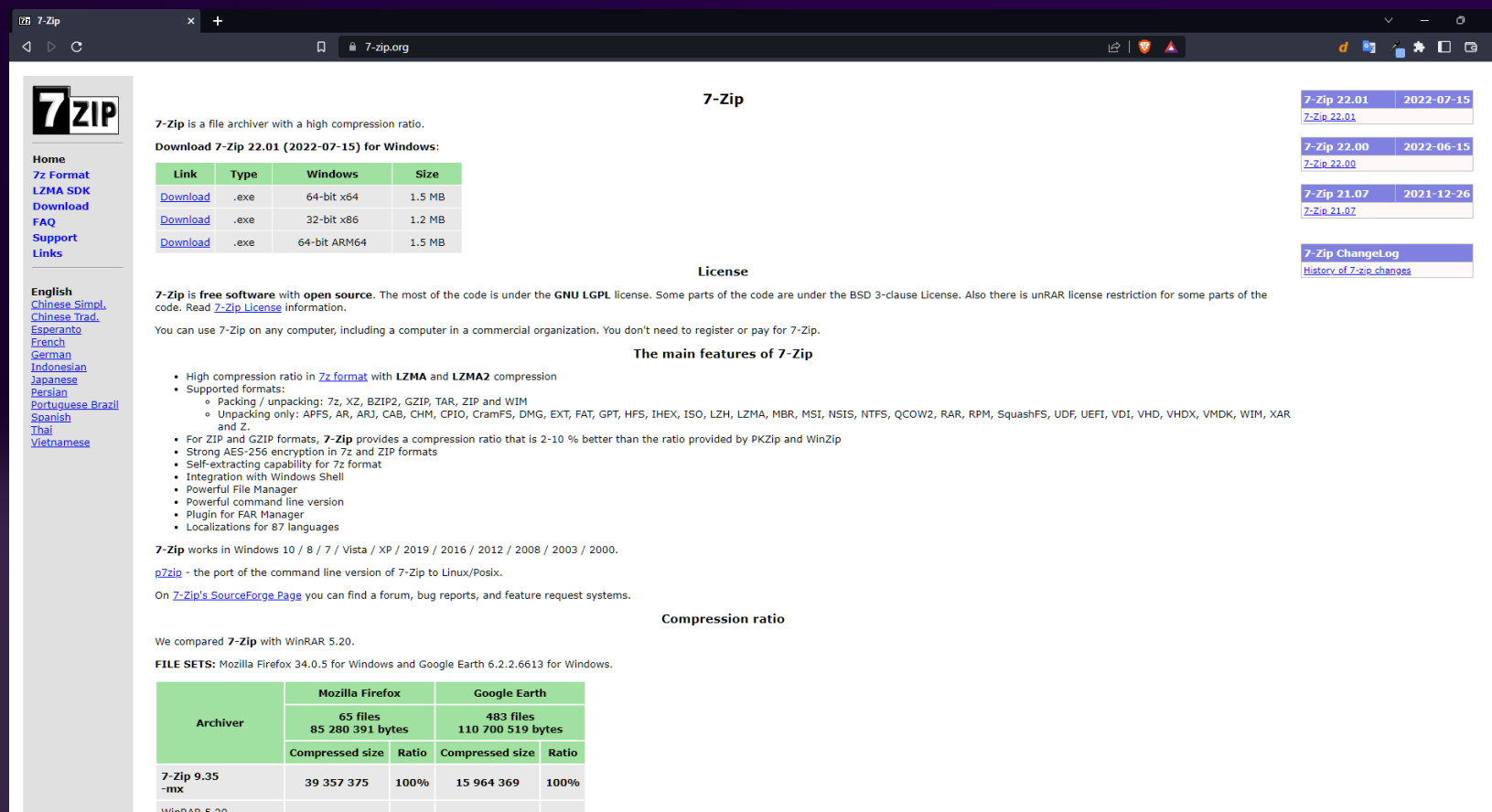

The screenshot shows the 7-Zip website in a browser window. The browser's address bar displays "7-zip.org". The website layout includes a sidebar on the left with navigation links, a main content area with a download table, and a right sidebar with version history and a changelog link.

7-Zip is a file archiver with a high compression ratio.

Download 7-Zip 22.01 (2022-07-15) for Windows:

Link	Type	Windows	Size
Download	.exe	64-bit x64	1.5 MB
Download	.exe	32-bit x86	1.2 MB
Download	.exe	64-bit ARM64	1.5 MB

License

7-Zip is **free software** with **open source**. The most of the code is under the **GNU LGPL** license. Some parts of the code are under the BSD 3-clause License. Also there is unRAR license restriction for some parts of the code. Read [7-Zip License](#) information.

You can use 7-Zip on any computer, including a computer in a commercial organization. You don't need to register or pay for 7-Zip.

The main features of 7-Zip

- High compression ratio in [7z format](#) with **LZMA** and **LZMA2** compression
- Supported formats:
 - Packing / unpacking: 7z, XZ, BZIP2, GZIP, TAR, ZIP and WIM
 - Unpacking only: APFS, AR, ARJ, CAB, CHM, CPIO, CramFS, DMG, EXT, FAT, GPT, HFS, IHX, ISO, LZH, LZMA, MBR, MSI, NSIS, NTFS, QCOW2, RAR, RPM, SquashFS, UDF, UEFI, VDI, VHD, VHDX, VMDK, WIM, XAR and Z.
- For ZIP and GZIP formats, **7-Zip** provides a compression ratio that is 2-10 % better than the ratio provided by PKZip and WinZip
- Strong AES-256 encryption in 7z and ZIP formats
- Self-extracting capability for 7z format
- Integration with Windows Shell
- Powerful File Manager
- Powerful command line version
- Plugin for FAR Manager
- Localizations for 87 languages

7-Zip works in Windows 10 / 8 / 7 / Vista / XP / 2019 / 2016 / 2012 / 2008 / 2003 / 2000.

[p7zip](#) - the port of the command line version of 7-Zip to Linux/Posix.

On [7-Zip's SourceForge Page](#) you can find a forum, bug reports, and feature request systems.


Compression ratio

We compared **7-Zip** with WinRAR 5.20.

FILE SETS: Mozilla Firefox 34.0.5 for Windows and Google Earth 6.2.2.6613 for Windows.

Archiver	Mozilla Firefox		Google Earth	
	65 files 85 280 391 bytes		483 files 110 700 519 bytes	
	Compressed size	Ratio	Compressed size	Ratio
7-Zip 9.35 -mx	39 357 375	100%	15 964 369	100%
WinRAR 5.20	41 388 543	105%	17 835 438	105%

DOWNLOAD THE 64-BIT EXE

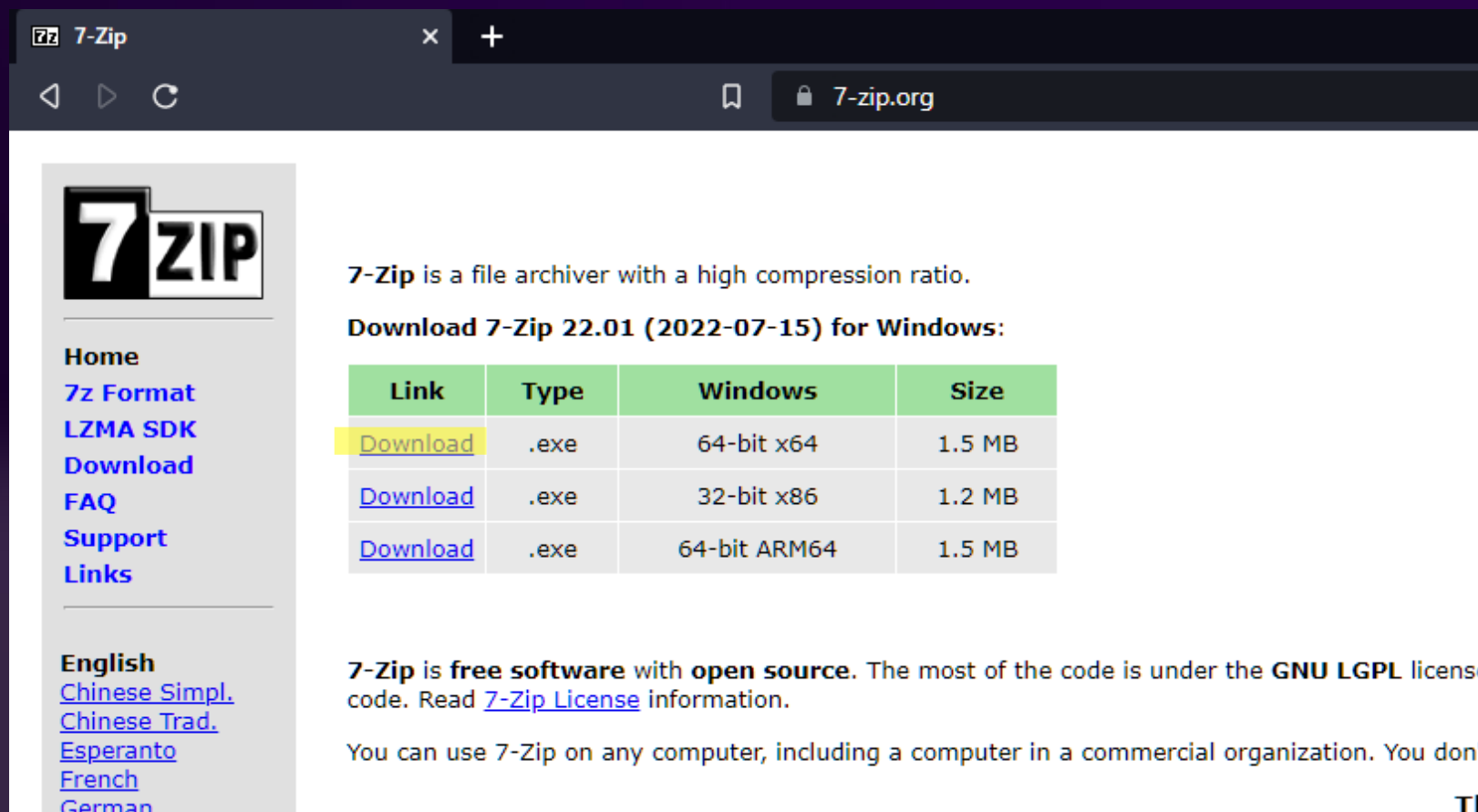

The screenshot shows a web browser window with the 7-Zip website. The browser's address bar shows '7-zip.org'. The website has a sidebar with navigation links: Home, 7z Format, LZMA SDK, Download, FAQ, Support, and Links. The main content area features the 7-Zip logo, a description of the software, and a table of download links for Windows. The table has four columns: Link, Type, Windows, and Size. The first row is highlighted in yellow, showing a 'Download' link for a 64-bit x64 .exe file, 1.5 MB in size. Below the table, there is a note about the software being free and open source, and a statement about its usage in commercial organizations.

7-Zip is a file archiver with a high compression ratio.


Download 7-Zip 22.01 (2022-07-15) for Windows:

Link	Type	Windows	Size
Download	.exe	64-bit x64	1.5 MB
Download	.exe	32-bit x86	1.2 MB
Download	.exe	64-bit ARM64	1.5 MB

7-Zip is **free software** with **open source**. The most of the code is under the **GNU LGPL** license code. Read [7-Zip License](#) information.

You can use 7-Zip on any computer, including a computer in a commercial organization. You don't


INSTALL THE 7ZIP PACKAGE

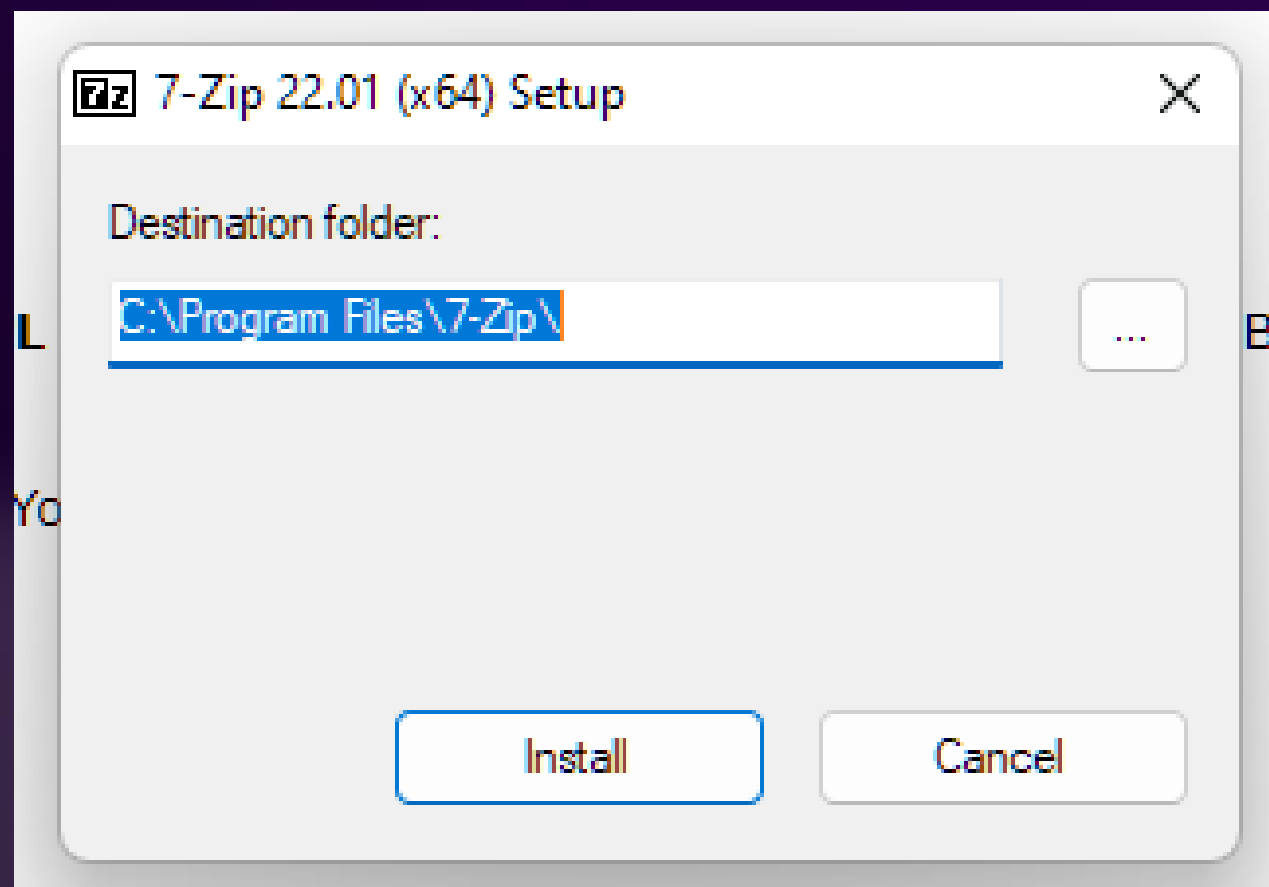




DOWNLOAD & INSTALL DENTAL SOFTWARE


NAVIGATE TO SHINING3DDENTAL.COM

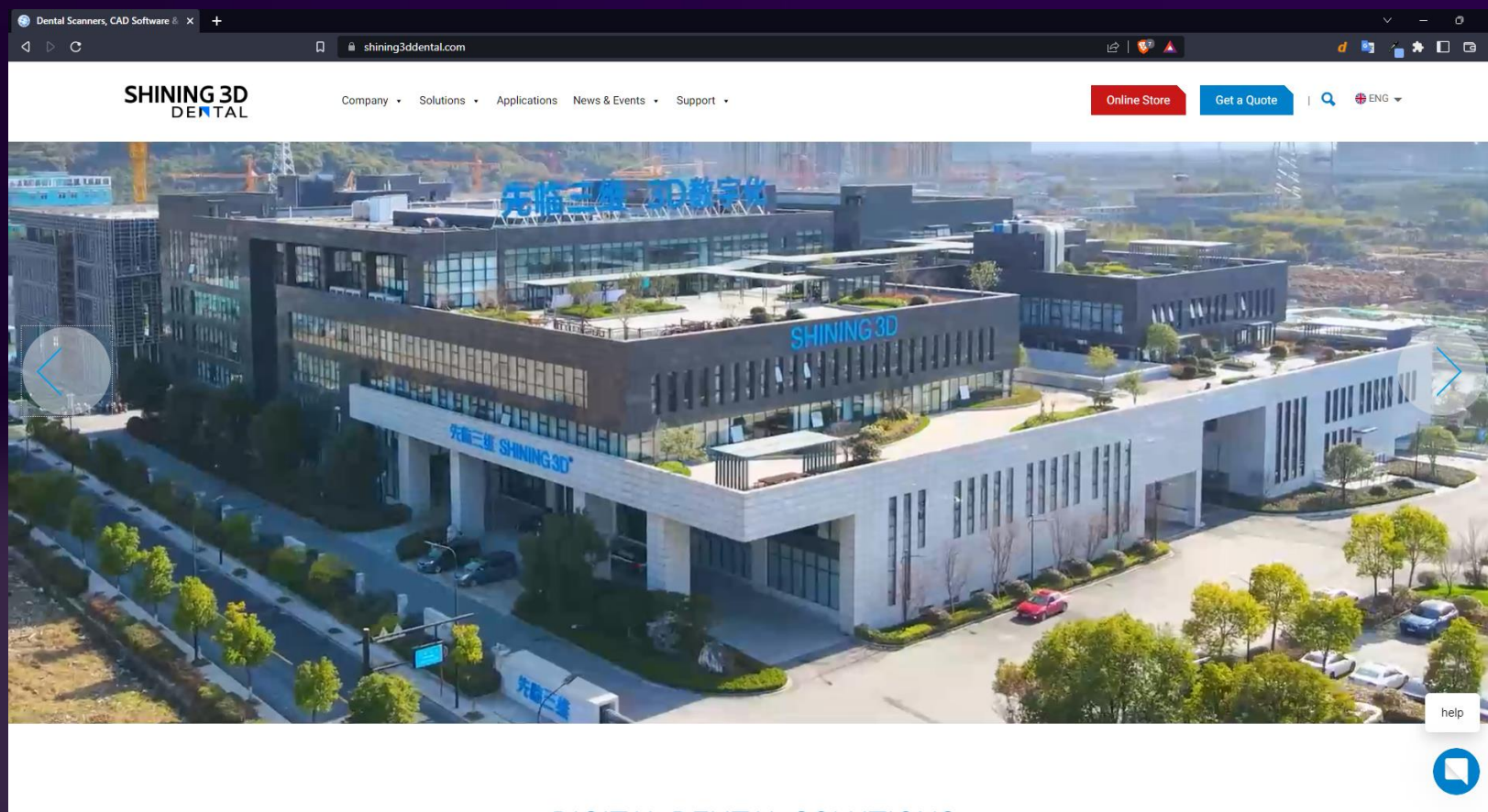


HOVER OVER THE SUPPORT TAB

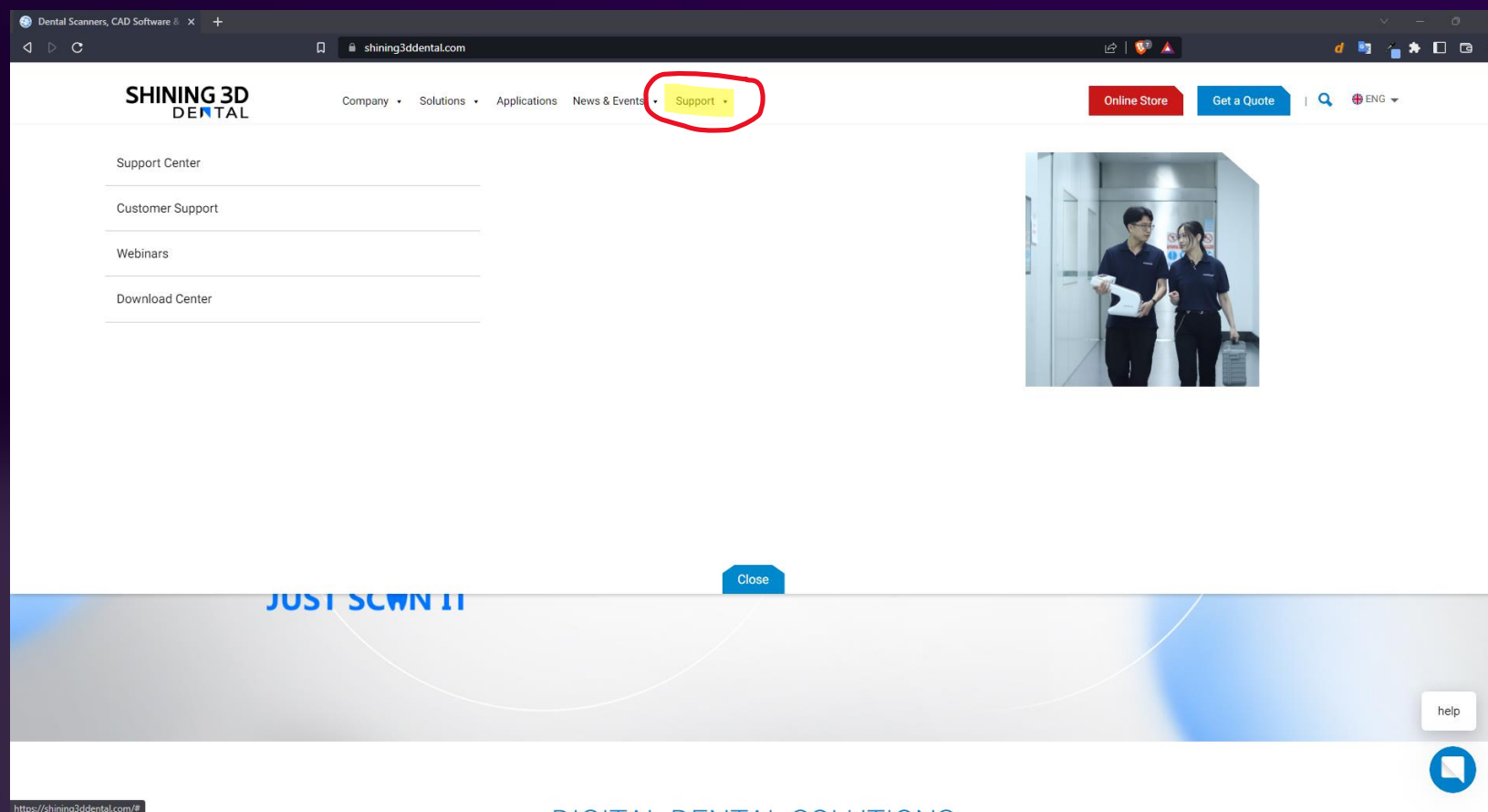


CLICK ON “SUPPORT CENTER”

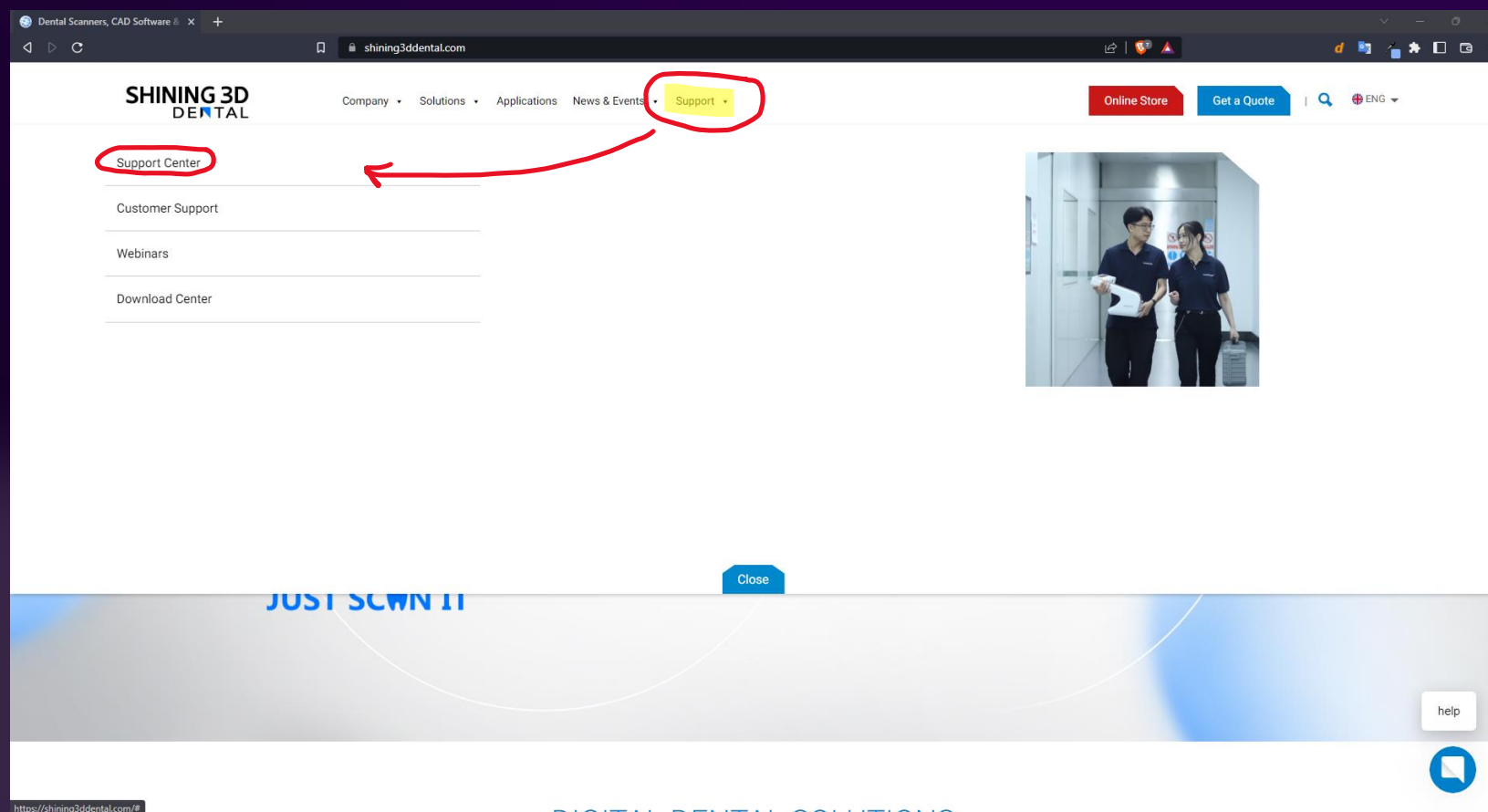


CLICK ON “DOWNLOAD CENTER”

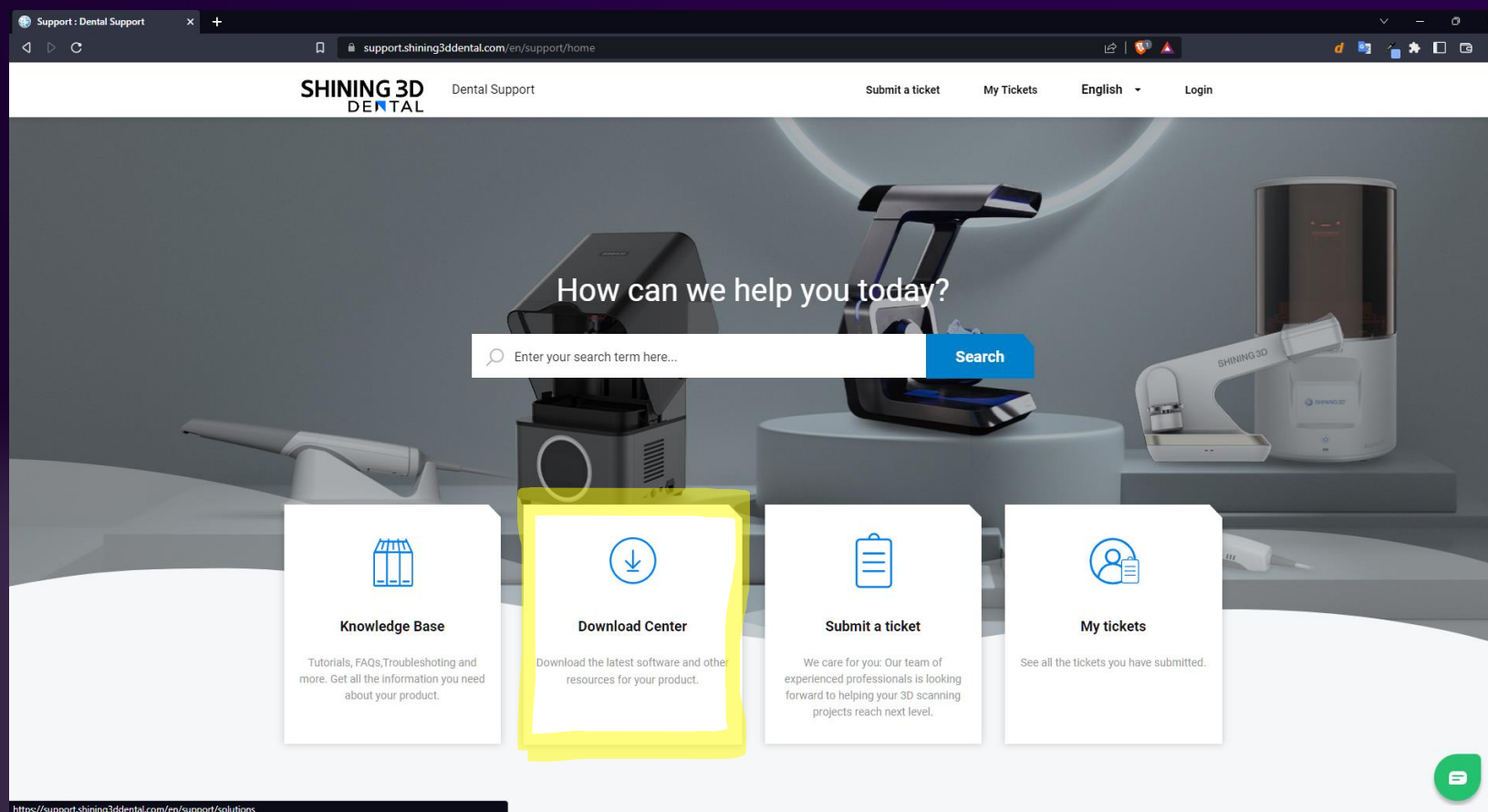

CLICK ON “DENTAL SOFTWARE”

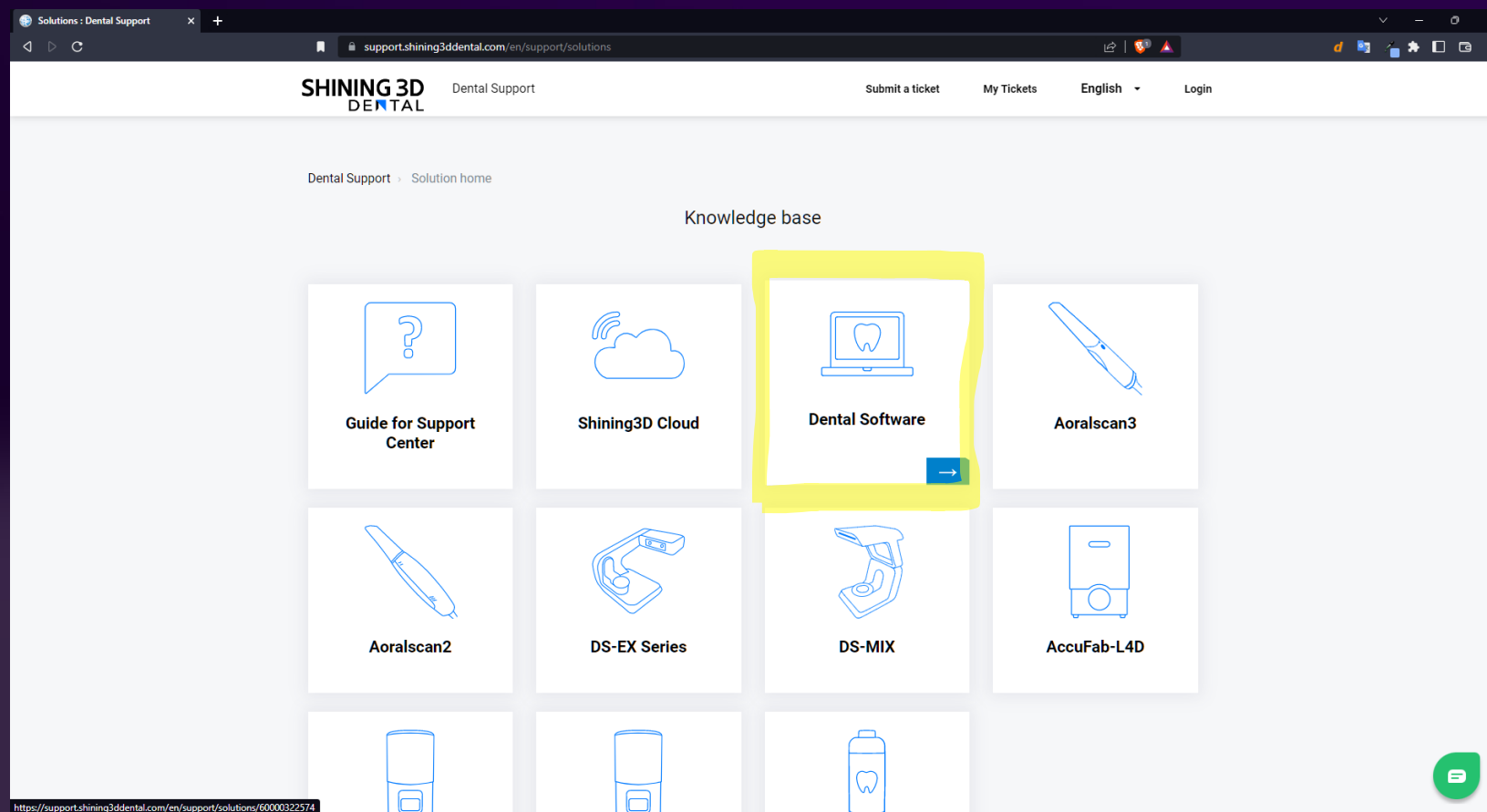

SELECT LINK FOR DESIRED SOFTWARE

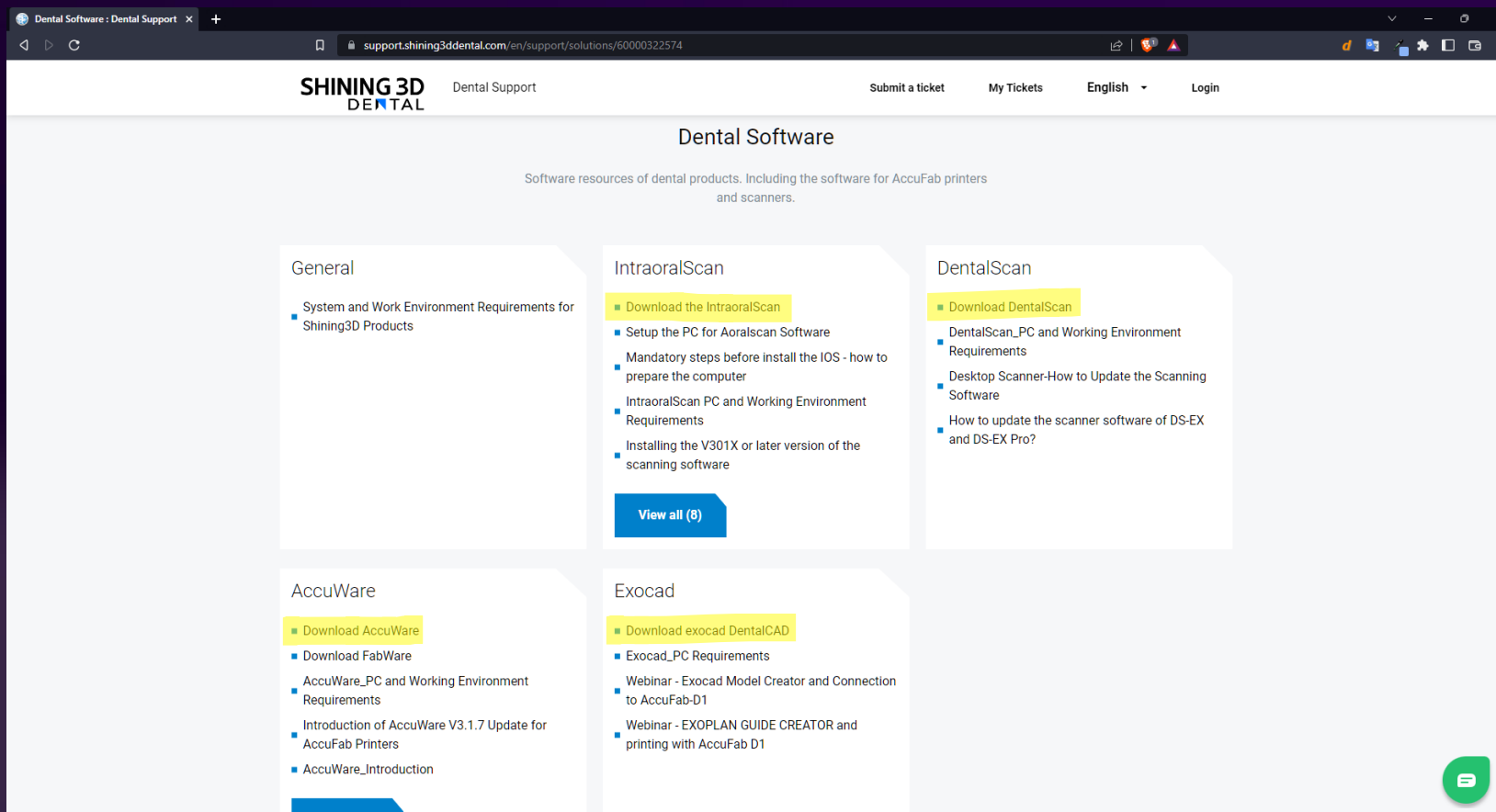

The screenshot shows a web browser window with the URL `support.shining3ddental.com/en/support/solutions/60000322574`. The page is titled "Dental Software" and features a navigation bar with "Submit a ticket", "My Tickets", "English", and "Login". The main content area is divided into four categories: General, IntraoralScan, DentalScan, AccuWare, and Exocad. Each category has a list of links and a "View all" button.

SHINING 3D DENTAL Dental Support

Submit a ticket My Tickets English Login

Dental Software

Software resources of dental products. Including the software for AccuFab printers and scanners.

General

- System and Work Environment Requirements for Shining3D Products

[View all \(8\)](#)

IntraoralScan

- [Download the IntraoralScan](#)
- Setup the PC for Aoralscan Software
- Mandatory steps before install the IOS - how to prepare the computer
- IntraoralScan PC and Working Environment Requirements
- Installing the V301X or later version of the scanning software

[View all \(8\)](#)

DentalScan

- [Download DentalScan](#)
- DentalScan_PC and Working Environment Requirements
- Desktop Scanner-How to Update the Scanning Software
- How to update the scanner software of DS-EX and DS-EX Pro?

AccuWare

- [Download AccuWare](#)
- Download FabWare
- AccuWare_PC and Working Environment Requirements
- Introduction of AccuWare V3.1.7 Update for AccuFab Printers
- AccuWare_Introduction


[View all \(6\)](#)

Exocad

- [Download exocad DentalCAD](#)
- Exocad_PC Requirements
- Webinar - Exocad Model Creator and Connection to AccuFab-D1
- Webinar - EXOPLAN GUIDE CREATOR and printing with AccuFab D1

[Chat](#)

ON THE RELEASE PAGE, CLICK DOWNLOAD LINK

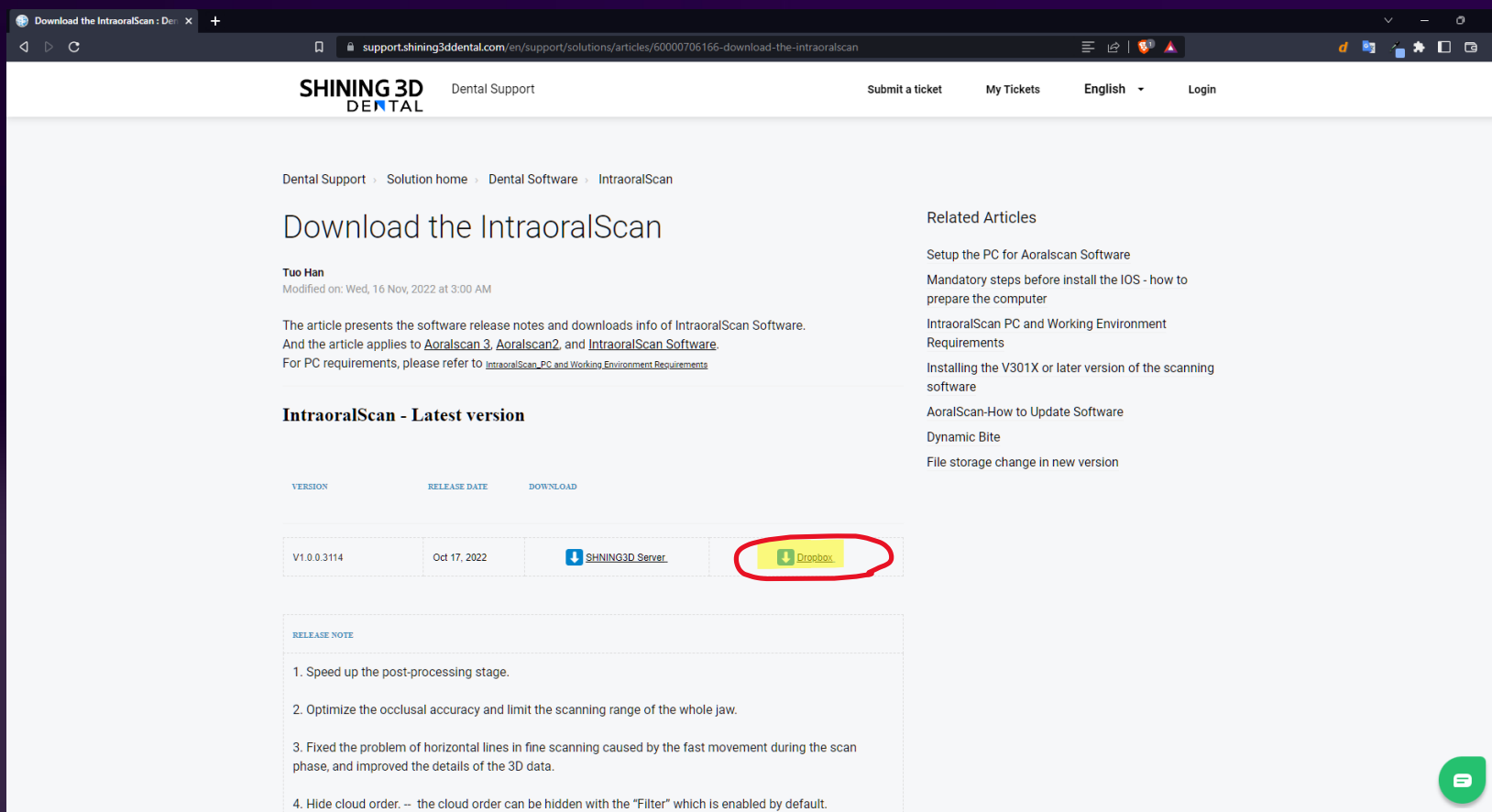

Download the IntraoralScan : Dental Support

support.shining3ddental.com/en/support/solutions/articles/60000706166-download-the-intraoralscan

SHINING 3D DENTAL Dental Support Submit a ticket My Tickets English Login

Dental Support > Solution home > Dental Software > IntraoralScan

Download the IntraoralScan

Tuo Han
Modified on: Wed, 16 Nov, 2022 at 3:00 AM

The article presents the software release notes and downloads info of IntraoralScan Software. And the article applies to [Aoralscan3](#), [Aoralscan2](#), and [IntraoralScan Software](#). For PC requirements, please refer to [IntraoralScan_PC and Working Environment Requirements](#)


IntraoralScan - Latest version

VERSION	RELEASE DATE	DOWNLOAD
V1.0.0.3114	Oct 17, 2022	SHINING3D Server Dropbox


RELEASE NOTE

- Speed up the post-processing stage.
- Optimize the occlusal accuracy and limit the scanning range of the whole jaw.
- Fixed the problem of horizontal lines in fine scanning caused by the fast movement during the scan phase, and improved the details of the 3D data.
- Hide cloud order. -- the cloud order can be hidden with the "Filter" which is enabled by default.


ONCE DOWNLOAD IS COMPLETE, PLEASE UNPACK FOLDER *WITH 7ZIP*

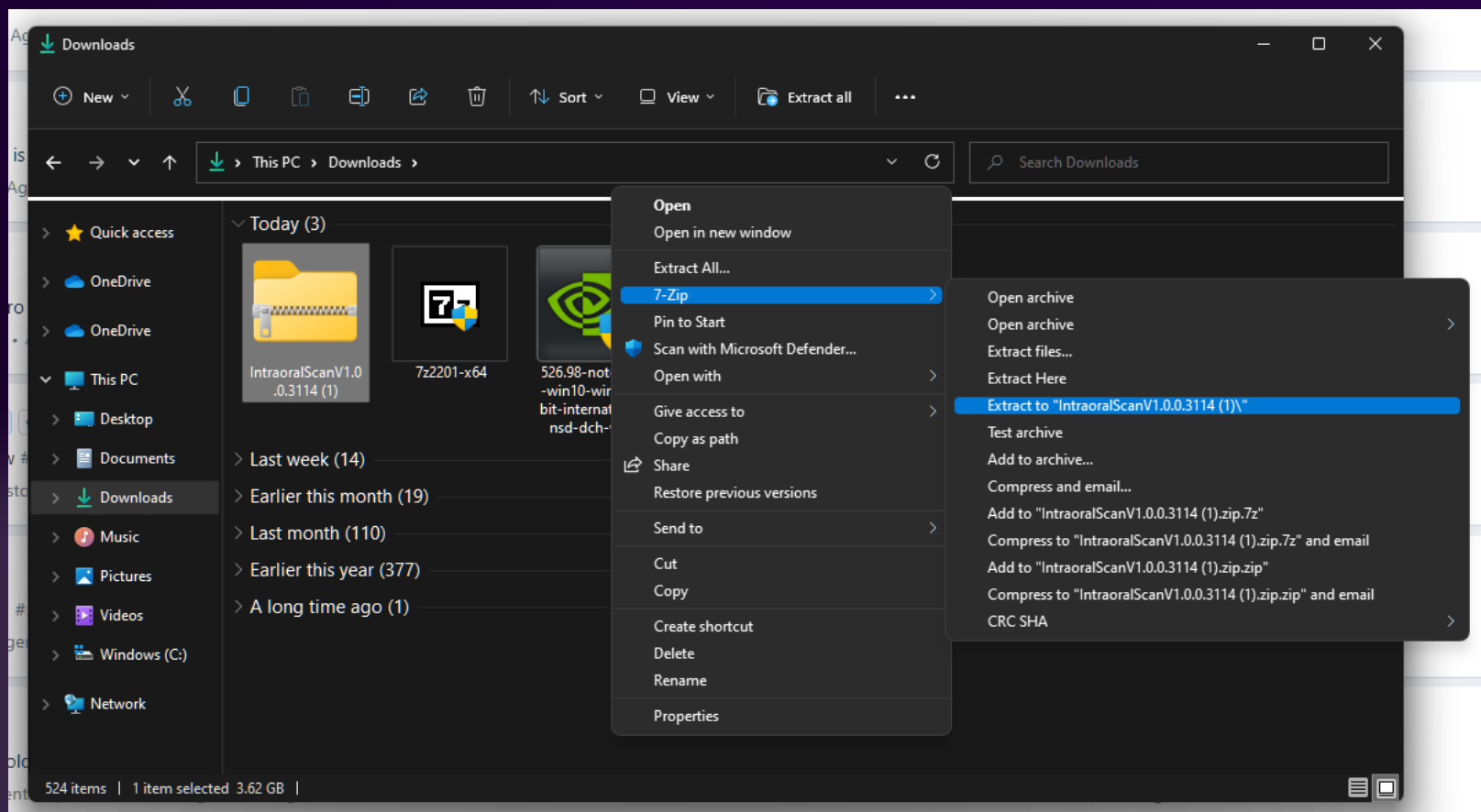


LOCATE THE UNZIPPED FOLDER & OPEN IT

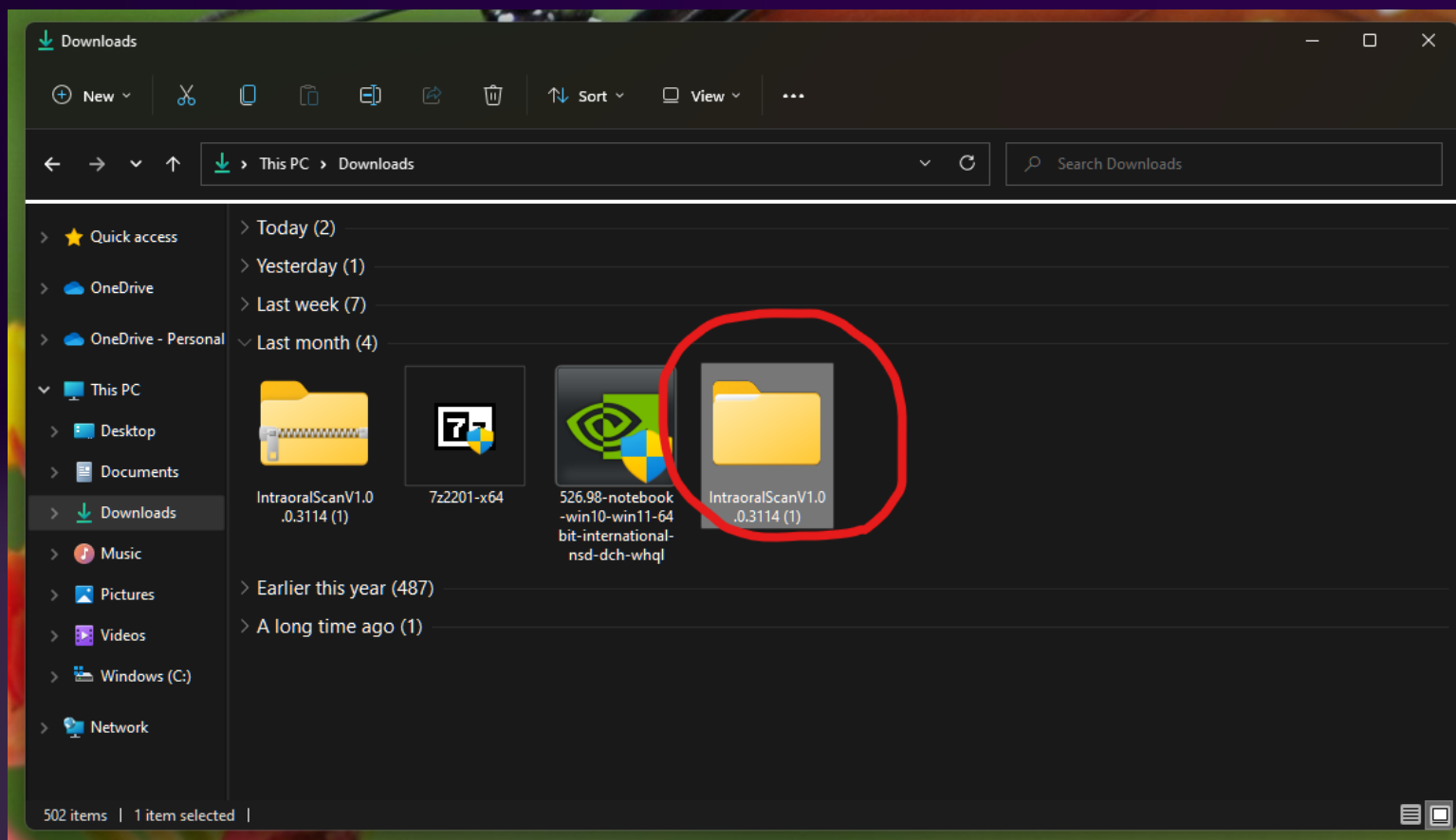


SELECT EXE FILE & RUN AS ADMINISTRATOR

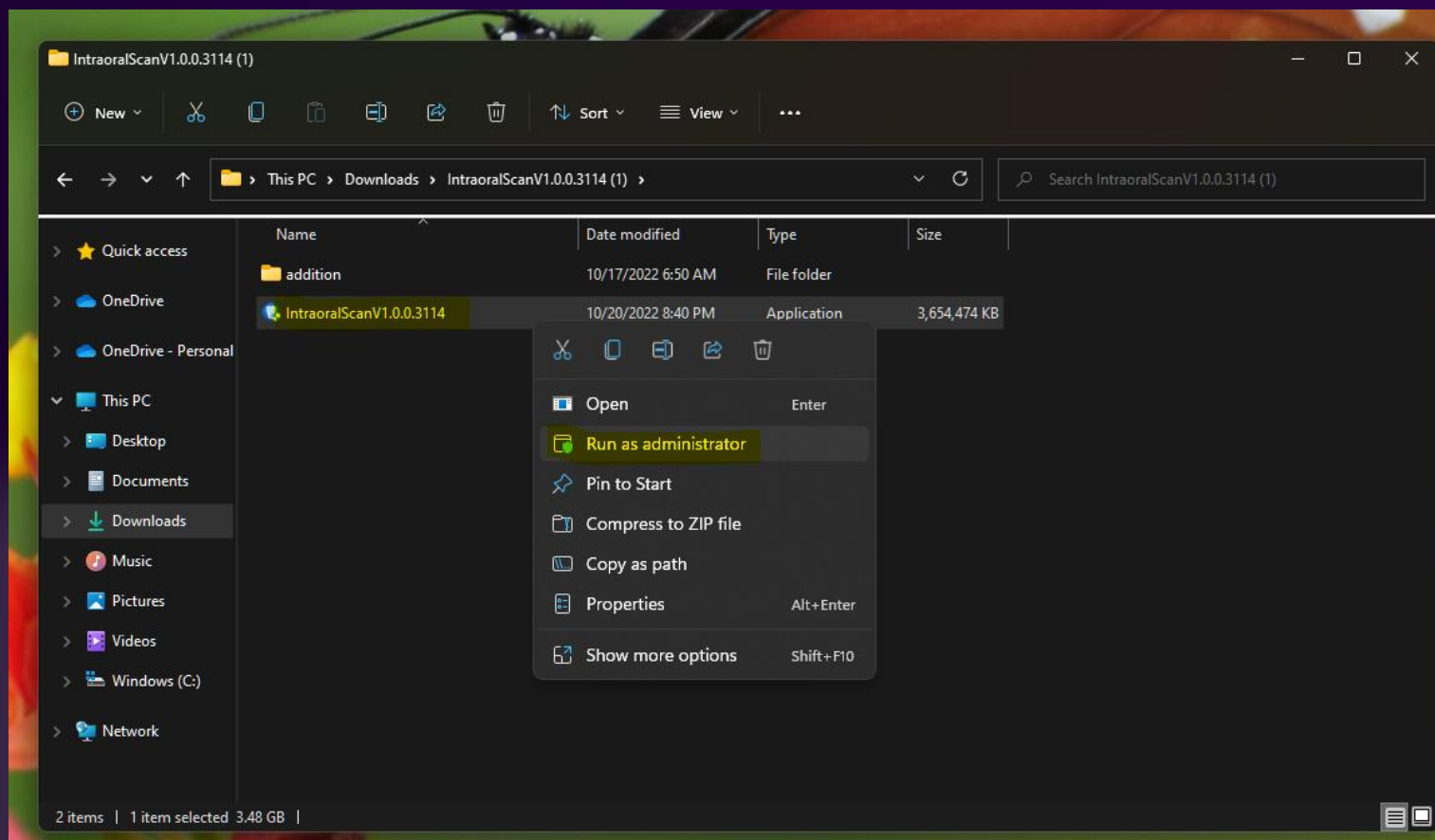


PROCEED WITH THE INSTALLATION PROCESS, INCLUDING UNINSTALLING AN OLDER COPY OF THE SOFTWARE, IF PRESENT

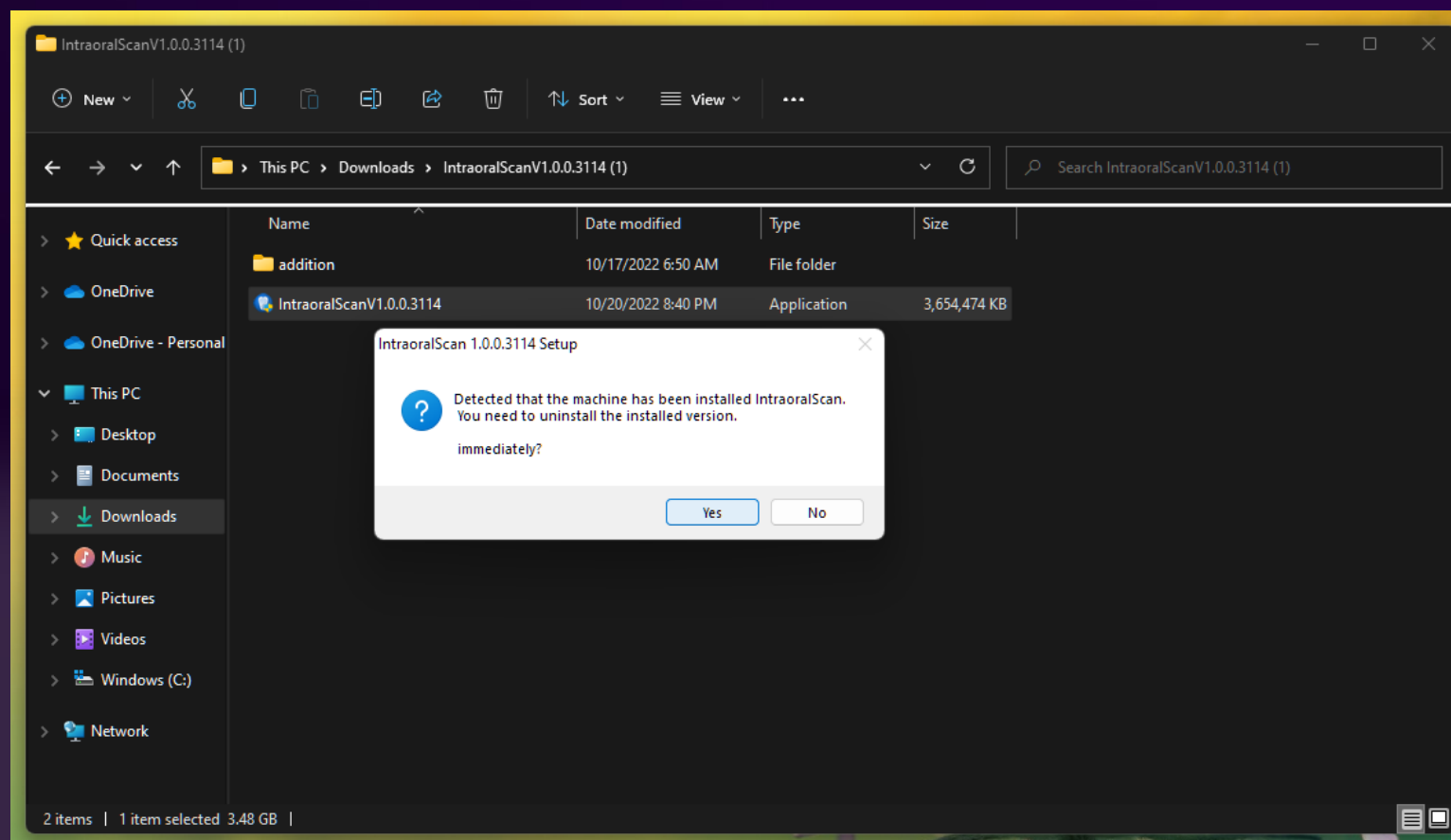


ENSURE THE CORRECT DEVICE IS SELECTED DURING INSTALLATION

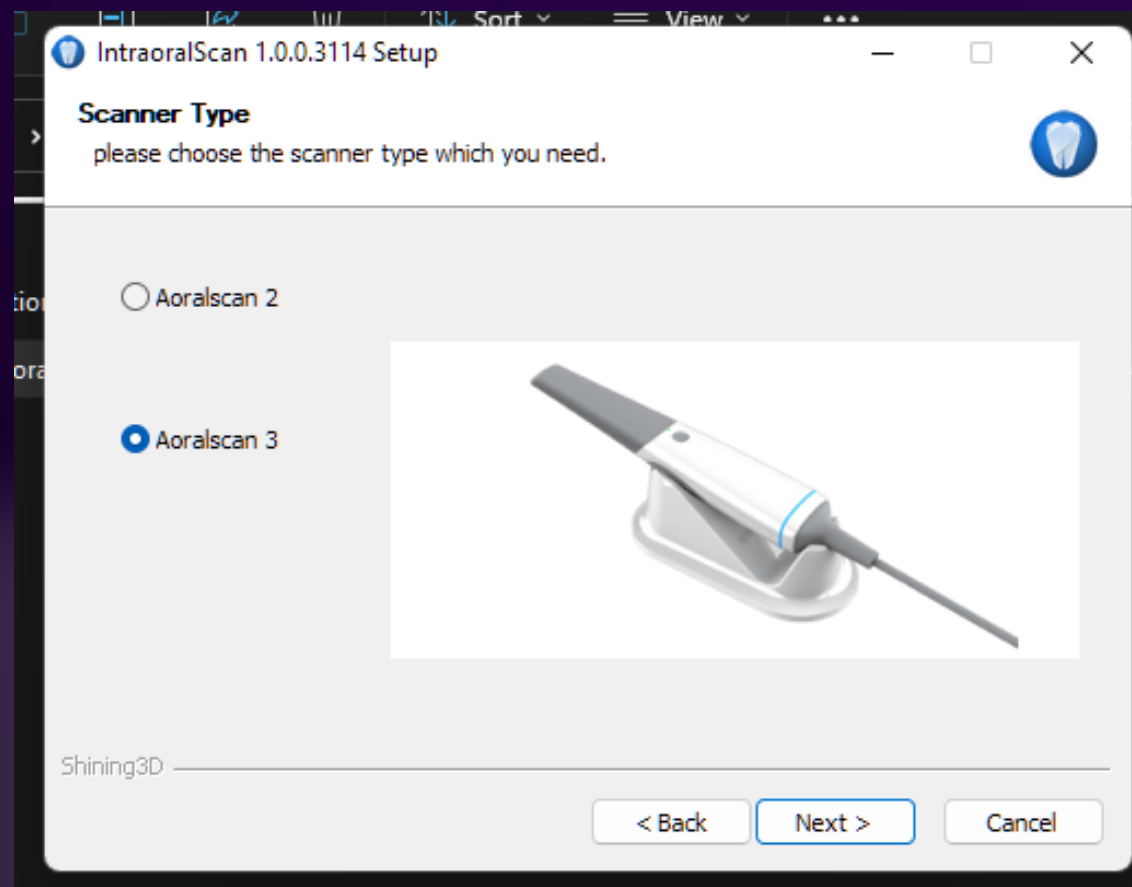


ALWAYS ENSURE THAT THE PROGRAM IS MAPPED TO THE (C:) DRIVE, NOT THE (D:) DRIVE OR ELSEWHERE

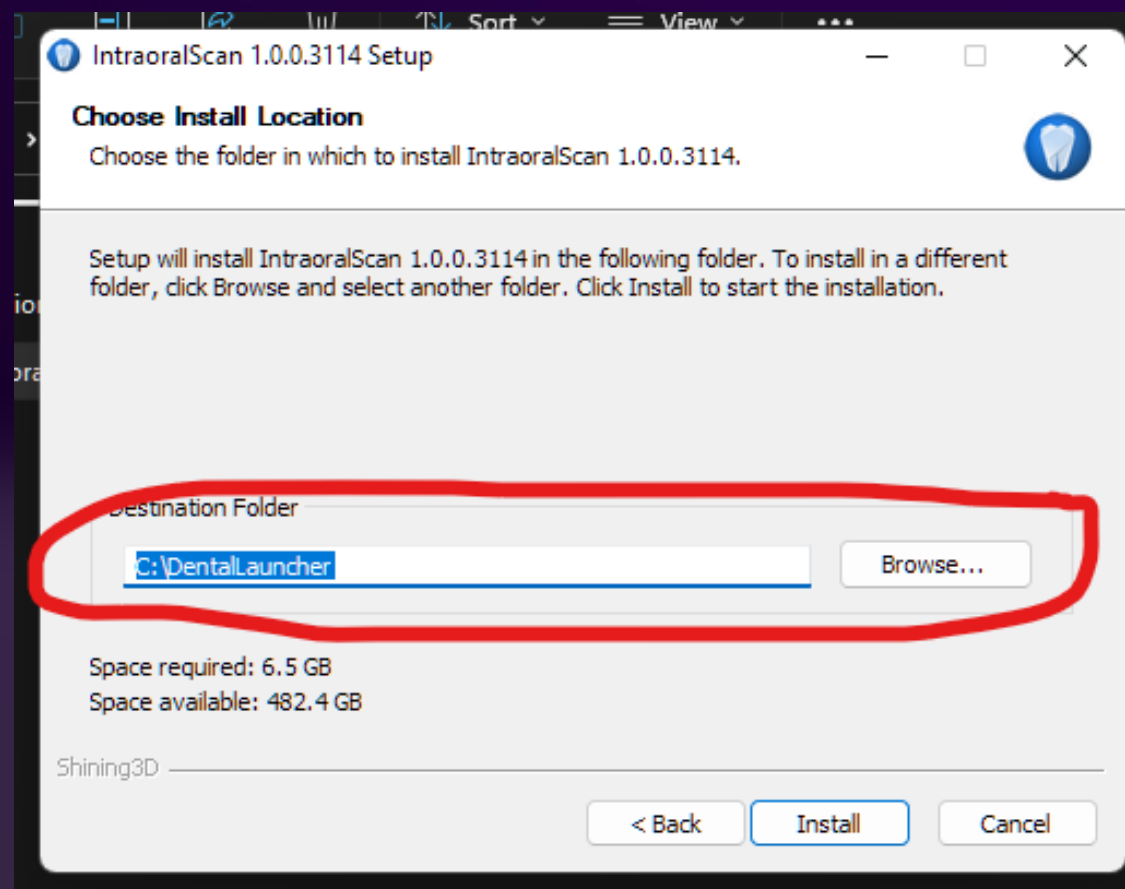


ALWAYS ENSURE THAT THE PROGRAM IS MAPPED TO THE (C:) DRIVE, NOT THE (D:) DRIVE OR ELSEWHERE

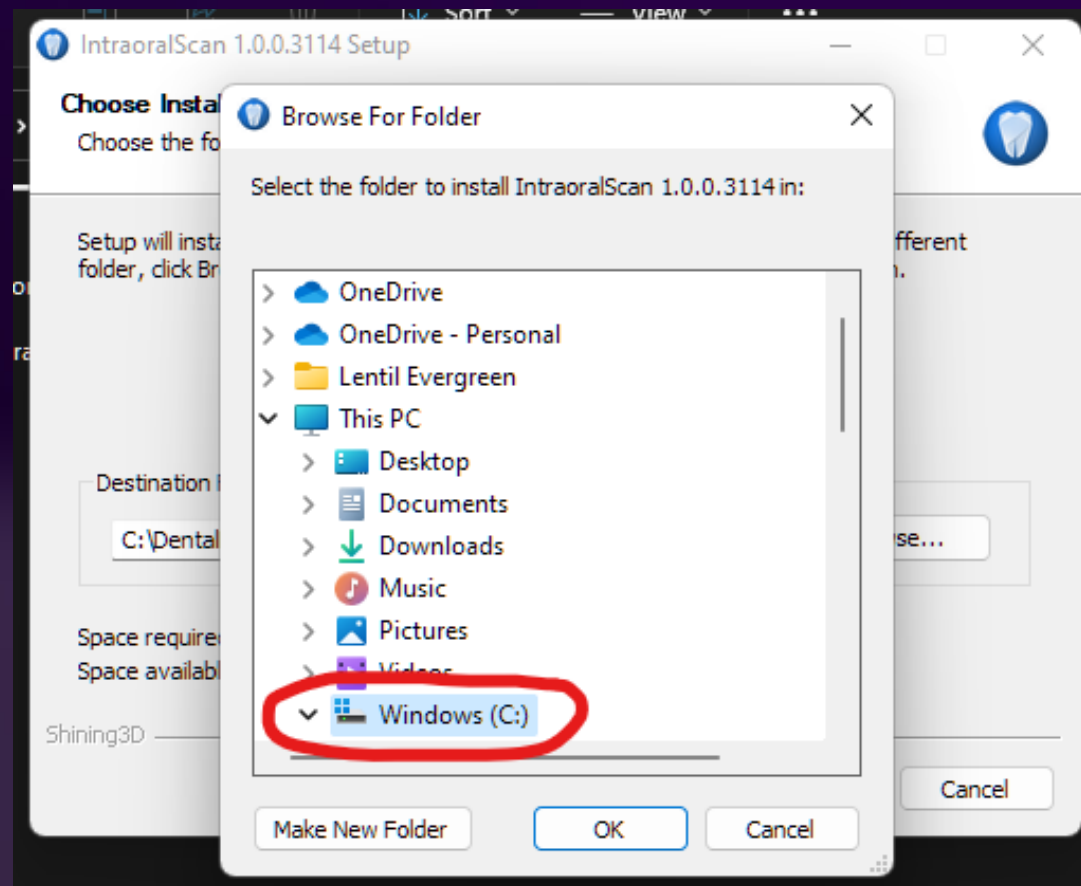


ALLOW THE INSTALLATION PROCESS TO COMPLETE

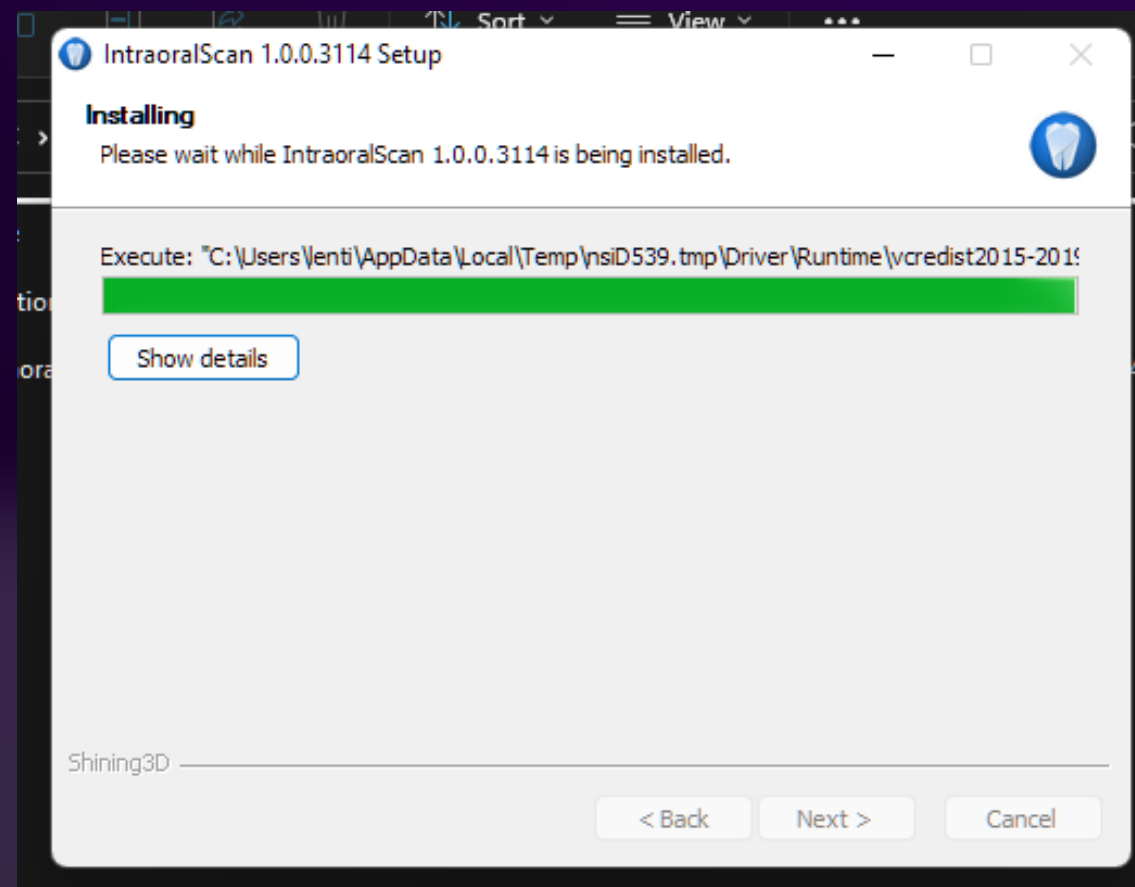

ALLOW THE INSTALLATION PROCESS TO COMPLETE

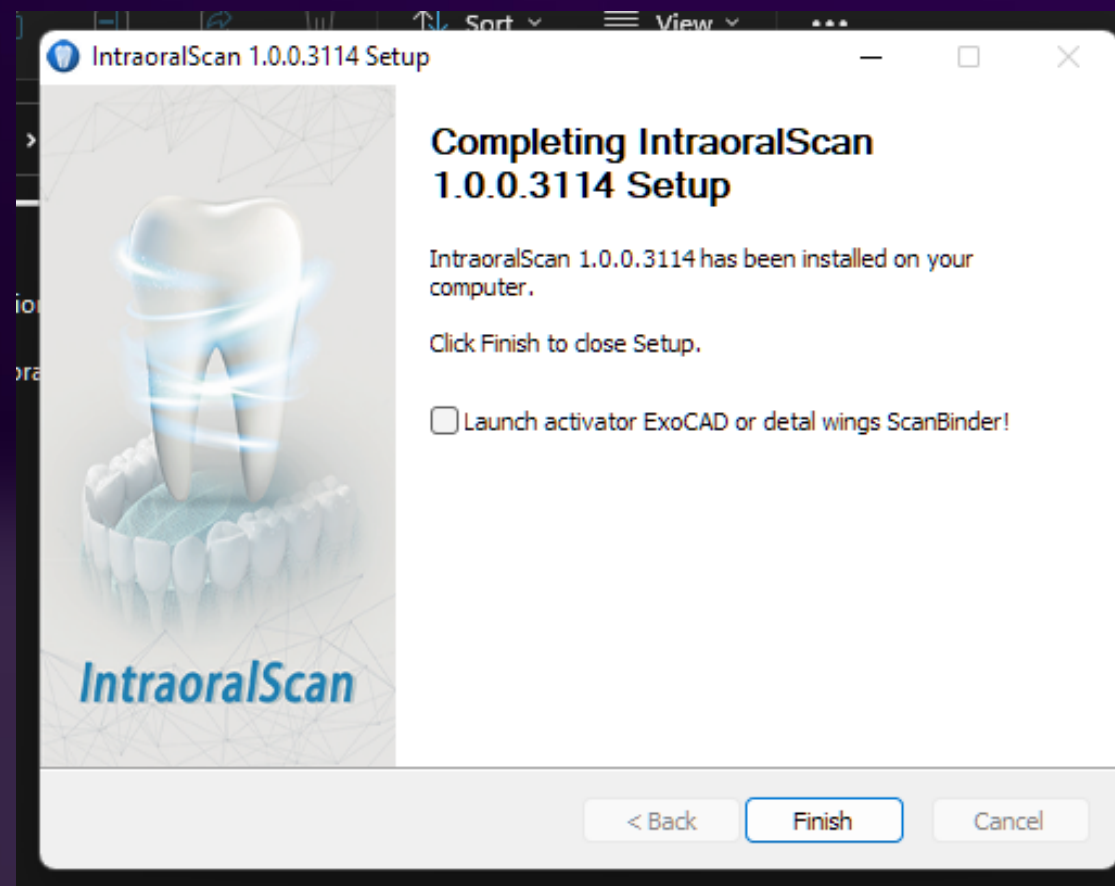

THE SOFTWARE IS NOW READY FOR DEVICE CALIBRATION & TESTING

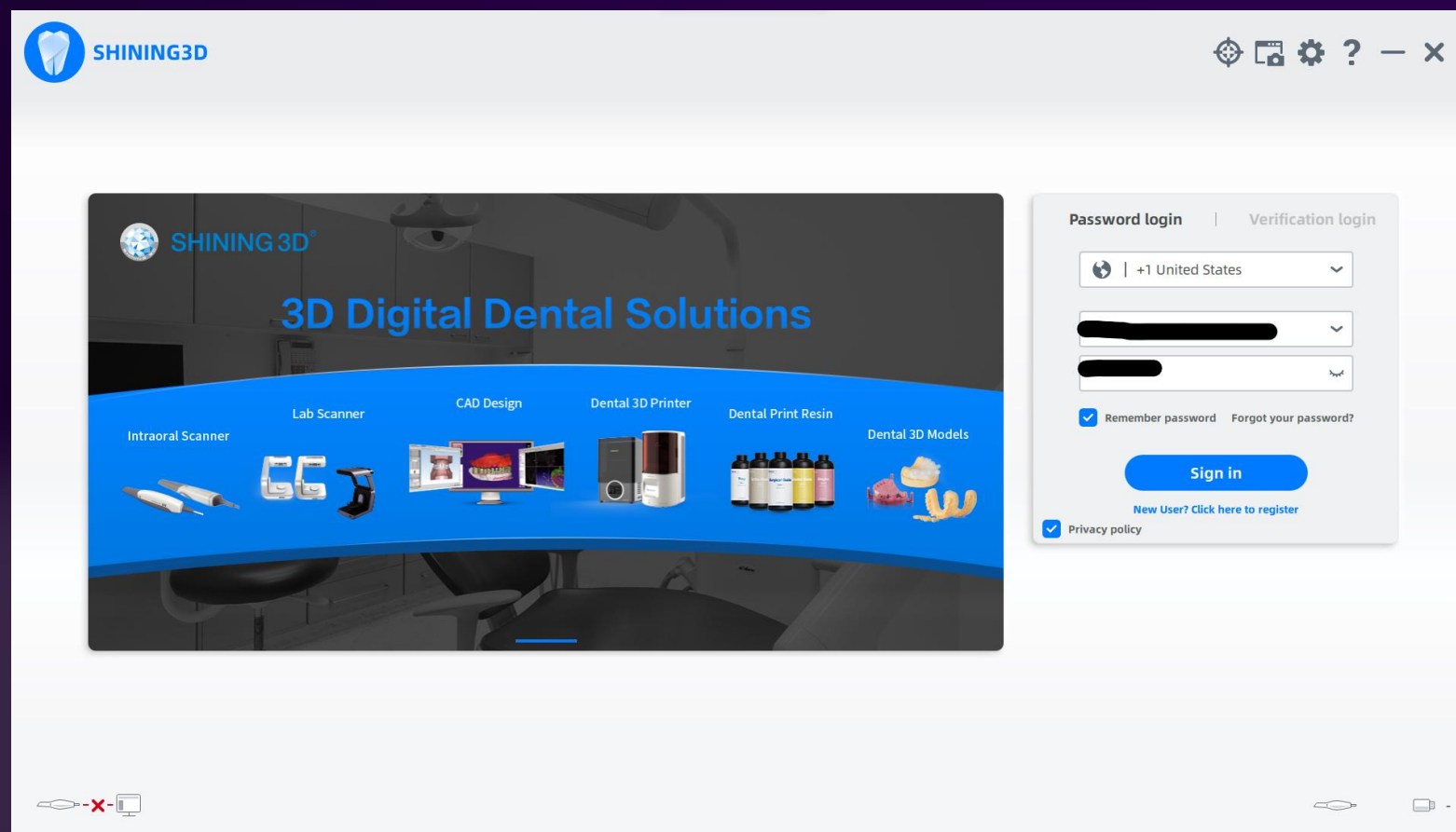

CHECK YOUR WORK:



STEP 1

- Ensure minimum system requirements are met




STEP 2

- Disable firewalls
- Uninstall all antivirus software




STEP 3

- Ensure GPU driver is up to date
- Ensure Windows OS is up to date

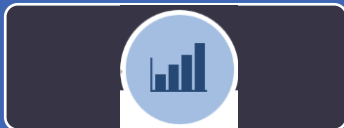

STEP 4

- Download 7zip
- Download dental software

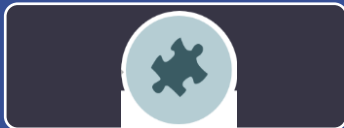

STEP 5

- Unpack & install software



STEP 6

- Test operation of software



SHINING 3D®

www.shining3ddental.com

GO DIGITAL WITH SHINING 3D



@shining3ddental

APAC Headquarters

SHINING 3D Tech. Co., Ltd.
Hangzhou, China
P: +86-571-82999050
Email: sales@shining3d.com
No. 1398, Xiangbin Road, Wenyan, Xiaoshan,
Hangzhou, Zhejiang, China,

EMEA Region

SHINING 3D Technology GmbH
Stuttgart, Germany
P: +49-711-28444089
Email: sales@shining3d.com
Panorama, Heilbronner straÙe 86, 70191,
Stuttgart, Germany

Americas Region

SHINING 3D Technology Inc.
San Francisco, United States
P: +1415-259-4787
Email: sales@shining3d.com
1740 César Chávez St. Unit D. San Francisco,
CA 94124



SHINING 3D®

THANKS!

APAC Headquarters

SHINING 3D Tech. Co., Ltd.
Hangzhou, China
P: +86-571-82999050
Email: sales@shining3d.com
No. 1398, Xiangbin Road, Wenyan, Xiaoshan,
Hangzhou, Zhejiang, China,

EMEA Region

SHINING 3D Technology GmbH
Stuttgart, Germany
P: +49-711-28444089
Email: sales@shining3d.com
Panorama, Heilbronner straÙe 86, 70191,
Stuttgart, Germany

Americas Region

SHINING 3D Technology Inc.
San Francisco, United States
P: +1415-259-4787
Email: sales@shining3d.com
1740 César Chávez St. Unit D. San Francisco,
CA 94124